- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

Zecken-Fix

de	Gebrauchsanweisung „Zecken-Fix“ Zeckenpinzette	4
en	User's Manual “Zecken-Fix” Tick Tweezers	9
fr	Mode d'emploi « Zecken-Fix » Pince à tiques	14
it	Istruzioni per l'uso “Zecken-Fix” Pinza per la rimozione di zecche	19
es	Instrucciones de empleo “Zecken-Fix” Pinzas para garrapatas	24
pt	Manual de operação „Zecken-Fix“ Pinça para carraças	29
ru	Руководство по применению «Zecken-Fix» пинцет для удаления клеща	34

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User's Manual Zecken-Fix tick removal tweezers

Dear Customer, thank you for choosing a KaWe product. Our products are known for high quality and durability. This KaWe device complies with the provisions of Regulation EU 2017/745 (European Regulation on Medical Devices), whereby it is classified as a medical product of class I.

Please carefully consider this instruction and read all the provisions prior to use. Follow given care instructions.

Please take into account: The use of Zecken-Fix tick removal tweezers may only be carried out in accordance with this User's Manual. Prior to use, follow the instructions for secondary treatment of the tick removal tweezers. Once a tick has been removed, the bite site must be disinfected immediately and, in any case, a doctor should be consulted as a precautionary measure. The tick removal does not protect against poisonous pathogens.

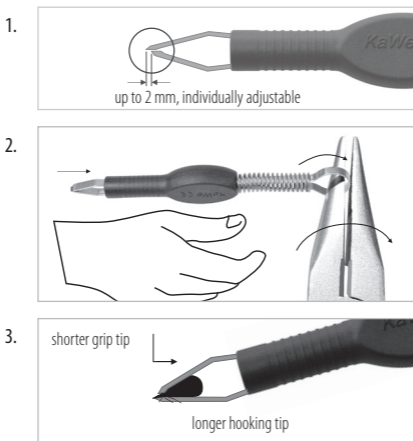
User: Medical staff and non-specialists (adults only!).

Target patient group: People and animals

Intended use: Tweezers for use in humans and animals to remove ticks.

Service:

Tweezers tip orientation: Tick removal tweezers are an instrument that facilitates the tick removal from the skin. The design of the Kirchner & Wilhelm tweezers allows the tweezers tips to be easily adjusted, giving each user the ability to adjust the tweezers to their individual application technique. This difference in the position of the tweezer tips is especially useful when the tweezers are applied at a slight angle to the skin

Before the start of the tick removal, one side of the tweezers is brought under the tick. At the same time, it is more convenient to use the instrument if the tips are slightly displaced relative to each other. If the tick can only be grasped from above, there is no need for displacement. Please note that the tip position of Kirchner & Wilhelm tweezers can be freely adjusted. Individual adjustment shall be carried out with conventional tongs.

Tick removal: The sooner the tick is removed, the lower the risk of transmission of viruses and bacteria.

en

Fig. 1

Grab the Zecken-Fix tweezers by the black handle and hold them between the thumb and middle finger. Put the index finger (of the same hand) on the annular end of the metal grip and pull the ring towards you to push the tick grip apart.

Fig. 2

Grab the tick with the tweezers tips as close to the skin as possible, slowly releasing the index finger. **At the same time, be careful not to crush the tick body!** Now slowly twist the tick (the direction does not matter). As a result of slow twisting, including pauses, the tick can crawl out on its own.

Improper use/contraindications to the use of the product: Any other use or use outside the above intended use shall be considered improper. The manufacturer is not responsible for damage resulting from such use. In this case, all risks are assumed solely by the user.

Additional information: if used and stored properly, the tick removal tweezers will serve you reliably for many years.

Warnings: Keep the tweezers out of the reach of small children. Danger of swallowing small parts! Before use, disinfect the Zecken-Fix tweezers with alcohol or clean with another disinfectant to eliminate (including the unlikely) infection with the tick removal tweezers. Tick removal should only be done by adults.

If, after attempting to remove the tick, part of its mouthpiece remains in the skin, please consult a doctor.

Medical recommendation: Tick bites can be associated with dangerous comorbidities. Therefore, in case of a tick bite, contact your doctor. Vaccination against spring-summer meningococcal meningitis is strongly recommended in known risk areas. In the event of a tick bite and the development of a wandering and spreading redness around it, consult a doctor immediately.

After the tick removal, the bite site must be disinfected. This can be done with alcohol-based disinfectants such as Sterillium®.

Secondary treatment limitations: In case of cracks or fragile places, the product must be disposed of.

Transportation and storage: Keep the instrument away from dust, humidity and contamination during storage and transportation. To prevent contamination and dust, store the tick removal tweezers only in its package.

Preparation for cleaning: Immediately after use, wipe off gross contamination with a wet wipe.

Manual cleaning/disinfection: Surface disinfectants or cleaners, such as pure alcohol or Hellipur.

Automatic cleaning: Before cleaning, the tweezers must be disassembled into their component parts (forceps, spring, handle).

- Cleaning: 55 °C / 10 min.
- Pre-cleaning and intermediate rinsing: 1 min.
- Validated by the cleaning and disinfection device: Miele PG 8582, rinsing technology: fresh water
- Recommended detergent: Dr. Weigert neodisher MediClean forte (dosage: 0.2% - 1.0%)

Automatic disinfection: Before cleaning/disinfection, the tweezers must be disassembled into their component parts (forceps, spring, handle).

- Cleaning: 55 °C / 10 min.
- Pre-cleaning and intermediate rinsing: 1 min.
- Disinfection: 90 °C / 5 min.
- A0 rated value: min. 3000 s
- Validated by the cleaning and disinfection device: Miele PG 8582, rinsing technology: fresh water / water without mineral salts
- Recommended detergent: Dr. Weigert neodisher MediClean forte (dosage: 0.2% - 1.0%)

Sterilization:

- Steam sterilization: 134 °C / 4 min.
- Drying time: 15 min.
- Validated by: Mocom Autoklav B22

Maintenance: No maintenance is required.

Functional control and check: Check the ease of movement of moving parts, for example, gripping legs. In case of damage or wear, the product must be disposed of.

General accessories: For more information on these products, please visit our home page: **www.kawemed.com**.

Disposal: The tweezers are made of stainless steel (grip + pressure spring) and plastic (handle). Usually, the product can be disposed of with household waste. Please observe the local regulations!

Warranty: If used properly and in accordance with the User's Manual, we provide a 2-year warranty from the date of sale. In case of questions, please contact your dealer!

Manufacturer: **KaWe**

Manufacturer's contact details: Dealer's address or phone number, please call:
+49-7141-68188-0.

Basic UDI-DI: 4030155KaWe1102Z2

Symbols:

	Manufacturer & date of manufacture		Please follow the User's Manual
	Article number		CE conformity mark
	Lot code		Attention!
	Medical device		UDI Data Carrier

Information for users and patients

All serious incidents involving this device must be immediately reported to the manufacturer and the competent authorities of the respective Member State in which the user and/or patient is located.

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KIRCHNER & WILHELM GmbH + Co. KG