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User manual · TEMDEX® Disinfectants and soap dispenser

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GB

Congratulations!

You bought an approved dispenser which is easy to handle and assures you working processes as used.

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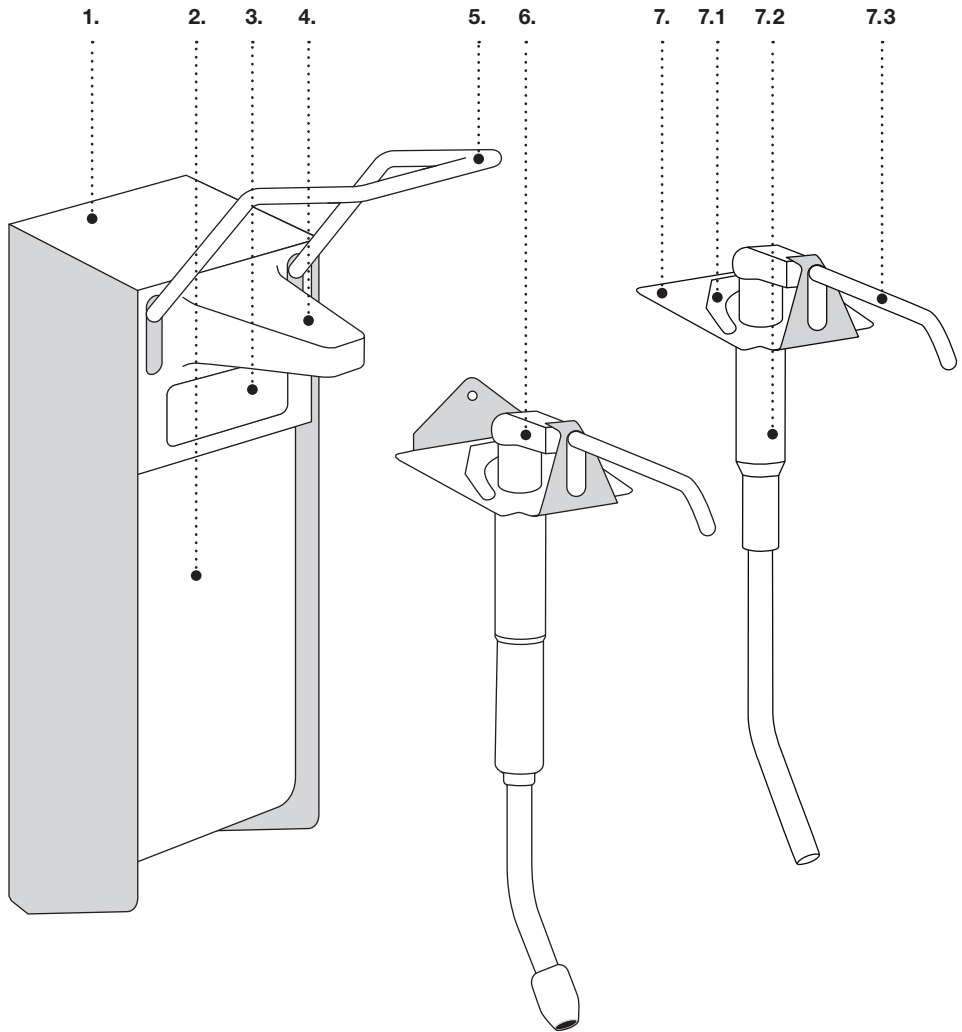
GB

1. Aluminium housing
2. Container
3. Marking clip
4. Outlet aperture
5. Stainless steel lever
6. Universal pump stainless steel
7. Universal pump plastic
- 7.1 Dosing fork
- 7.2 Suction tube
- 7.3 Spout

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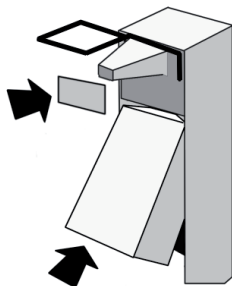
GB

1. Mounting of Dispenser

Please identify the correct position of the dispenser before it is mounted. The following minimum clearances must be observed:

- Above the dispense:
keep a space of minimum 40 mm
- Below the dispenser:
min. 150 mm for 500 ml dispenser
min. 200 mm for 1000 ml dispenser
- Lift up bottle lightly and pull it downward out of the body.
- Press locking on the wall plate and pull wall plate downward out of the body.
- Insert dowels in predrilled holes and fix the wall holder with screws.
- Push dispenser body top down over wall holder until you hear the locking snap in.
- Push full bottle bottom-up over the dispenser pump and insert in the body.
- When first used press down the lever several time until liquid leaks from the pump head. Catch this drop with a convenient pot.
- The dispenser is ready for use now.

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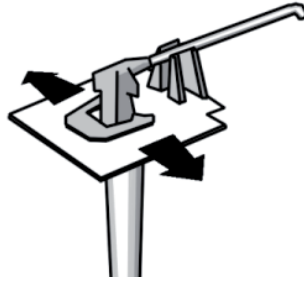
2. Important notice

Dosing of liquid:

Delivery is with a max. volume of 1,8 ml per pass. By changing the dosing fork you can reduce the volume.

By pushing right: approx. 1,2 ml per pass.

By pushing left: approx. 0,8 ml per pass.

Autoclaving the dispenser:

Proceed as described in step 1 and 2. Autoclave the dispenser using hot steam (121°C at a pressure of 1 bar).

Attention: The pure plastic pump is not autoclavable.

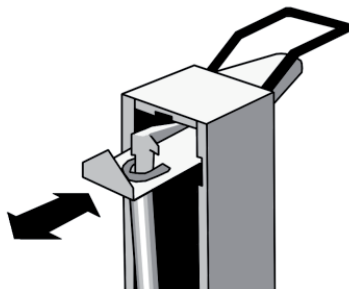
Maintenance:

To ensure proper function of the dispenser and to ensure the basis of recommendations for infection prevention and hygiene, the dispenser and the pump should be regularly cleaned and if necessary disinfected.

- For cleaning the pump first remove bottle, then press locking at the wall plate and pull dispenser upwards.
- Hold lever in upper position and pull out the pump backwards. For inserting again go reverse.
- Use gentle cleaning agents only.
- If the pump is blocked (e.g. no use for a longer time), remove bottle and rinse thoroughly with hot water until the work is perfect again.

Note: Plastic pumps are spare and wear parts and should be replaced at an interval of six

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TEMCA GmbH & Co. KG