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Instructions for use

Indications

- Tension
- Muscular discomfort
- Joint complaints/arthritis
- Rheumatic pain

Do not use for arterial circulation problems, cardiovascular diseases and acroparaesthesia.

In case of any questions, please consult your doctor before use.

Heating in the microwave

- Heat the product approx. 1 – 3 min. at 600 watt
- Please heat in 30-second intervals until the desired temperature has been reached
- If your microwave has a rotary plate, please ensure that it can turn freely

Heating in the oven

- Heat the product approx. 10 – 12 min. at 100 °C
- When heating, always place a sheet of baking paper underneath the product to avoid burning

Only for intended use!

Important safety instructions

- Do not leave the product unattended when heating
- In order to prevent burns, please check the temperature before applying
- Its use is intended to support the therapy and does not replace a visit to the doctor

Materials

- Filling and protective cover: see laundry label

Care and cleaning instructions

- Always store in a dry place after use
- Heat a completely dry product only
- The product is not washable, but its cover can be wiped clean with a damp cloth when necessary
- Additionally place a cup of water in the microwave or oven every 10th heating process in order to prevent the filling from drying out

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