
- Ⓓ **Montageanleitung- und Gebrauchsanleitung**
- ⒼⒸ **Installation and Operating Instructions**
- Ⓕ **Notice de montage et mode d'emploi**
- ⒼⒻ **Installatie-en gebruikshandleiding**
- Ⓘ **Istruzioni di montaggio e per l'uso**
- Ⓔ **Instrucciones de instalación y funcionamiento**
- ⒴ **Instrukcja montażu i obsługi**

Artikel-Nr.  
Item number  
No de référence  
Type Nr.  
Nr. Articolo  
N° de artículo  
Numer produktu

3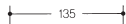

Zweckbestimmung  
Proper use  
Affectation  
Toepassingen  
Conformità di norma  
Usò previsto  
Przeznaczenie

4

Warnhinweise  
Warning  
Mises en garde  
Let op  
Avvertenze importanti  
Advertencias  
Ostrzeżenia

5 - 8

Wartung  
Maintenance  
Entretien  
Onderhoud  
Manutenzione  
Mantenimiento  
Konservacja

9 - 12


Pflegehinweise  
Care tips  
Conseils d'entretien  
Onderhoudsinstructies  
Consigli per la pulizia  
Indicaciones para el mantenimiento  
Pielęgnacja

13 - 15

**Duschhocker**  
**Shower stool**  
**Tabouret de douche**  
**Douchekruk**  
**Sgabello da doccia**  
**Taburete para la ducha**  
**Stołek prysznicowy**

802.51.30092  
802.51.30097  
802.51.31092  
802.51.31097  
802.51.W300  
802.51.W310

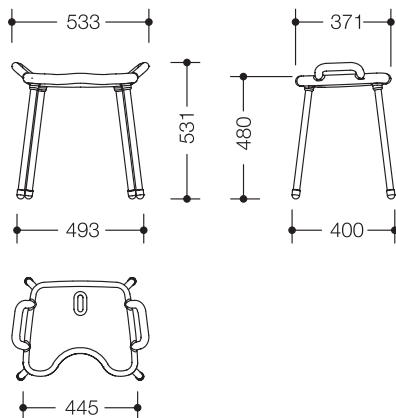CE

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## WARNING!

### Use of HEWI shower stool

The HEWI shower stool is designed to withstand a maximum static working load of 150 kg. Do not use this stool for exceeding loads! Any non-compliance with loading limits can cause permanent damage to the stool. In such cases, the stool must be examined by the company appointed to carry out maintenance work prior to the shelf being used again. If in doubt, contact HEWI's after-sales service team on tel. +44 1634 377688 for further assistance.

Only use the HEWI shower stool as described in the instructions. Any improper use, e.g. as a gymnastic apparatus or rocking with it, can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the shower stool is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into it. The fact that the stool bends to a certain extent under load is normal. Should the stool become permanently deformed, however, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service on tel. +44 1634 377688.

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### **Maintenance of shower stool**

In purchasing the HEWI shower stool, you have chosen a high-quality product. Once assembled, the stool requires no special maintenance.

If any problems occur, you should immediately contact HEWI's after-sales service team on tel. +44 1634 377688 for further assistance.

### **Damage in transit**

The shower stool leaves the factory in perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to theseat functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the seat is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the seat in its packaging whenever possible. Once the seat has been installed, put the protective foil back around it until the support needs to be used.

### **Exclusion of liability**

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's seat with lift-up feature, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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**GB**

### **Care tips**

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough.

If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a pH-rating of between 6 and 8. Detergents containing active substances such as phosphates, soap and tensides may be used.

Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling.

**For further information please contact HEWI, tel. +44 1634 37688.**

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HEWI Heinrich Wilke GmbH