
Etac Swift Mobile

Manual


etac®

78560D 16-10-03

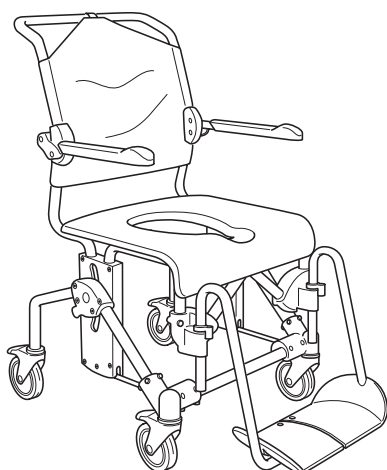
Swift Mobile

Swift Mobile 160
Swift Mobile 160S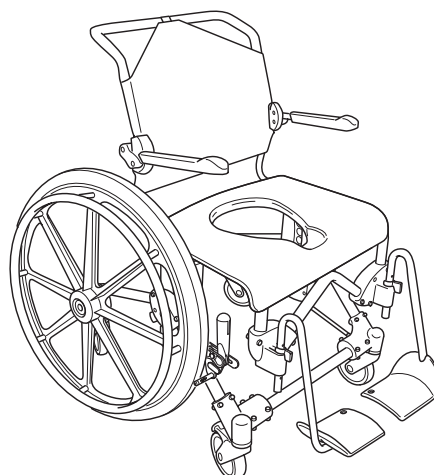

Swift Mobile 24"

Swift Mobile 24" 160

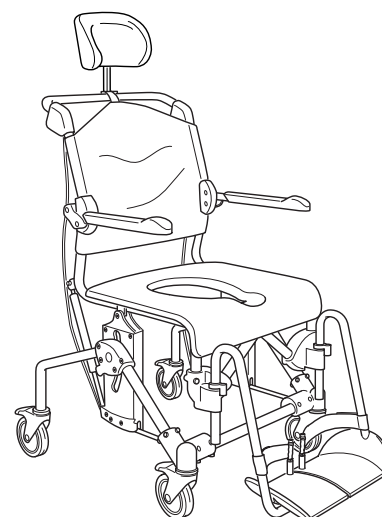Swift Mobile Tilt

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English

General

Thank you for choosing a shower chair from Etac.
In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair.
The assistant is the person operating the chair.

The Swift Mobile family is a range of mobile shower and toilet chairs intended for use when carrying out hygiene care for people with decreased mobility. The shower chair can be used in showers, at sinks or over the toilet, as well as for transfers to and from the hygiene room.

Swift Mobil has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

The images which are referred to on each page are found at the start of the manual (pages 3 to 9). See Figures A to J for the corresponding text.

This product should be recycled as per national regulations.

Item list: ● = Applies to, ○ = Accessories

See the item list on the right which displays the information applicable to each product.

This manual covers the following products:

Swift Mobile, Swift mobile 160 & Swift Mobile 160 S	●		
Swift Mobile 24", Swift mobile 24"160		●	
Swift Mobile Tilt, Swift mobile Tilt 160 & Swift mobile Tilt 160 S			●

Swift Mobile
Swift Mobile 24"
Swift Mobile Tilt

Product description

See Figure A

A1. Bolt, **A2.** Legrest, **A3.** Seat, **A4.** Armrest, **A5.** Backrest upholstery

A6. Headrest

● ● ●
○ ○ ●

Assembly

See Figure B

● ● ●

Dimension table

See Figure C

● ● ●

Warranty / maintenance

See Figure D

● ● ●

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.
Service life 7 years. For complete information regarding the service life of the product, see www.etac.com.

your shower and toilet chair is not in perfect working order, contact your dealer.

Maintenance and reconditioning

To keep the chair in good and safe working order, you should regularly:

- D1.** Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- D2.** The product can be cleaned at 85 °C for 3 minutes.
- Keep the castors free of hair and soap deposits, etc. If

Safety

See Figure E

● ● ●

- Defective chairs must not be used.

E1. Make sure the castors are locked when users sit down on or get up from the chair.

- An anti-tip attachment must be used if the user is an amputee.

- E2.** Never stand on the foot supports as there is a risk of tipping!
- E3.** The sitting position has an impact on stability.
- E4.** Chair stability is improved if the castors point outward.
- E5.** See the maximum user weight for your commode chair in the table (page 5).

Transfers

See Figure F

● ● ●

Over a threshold:

- F1.** Chair only, put one foot on the chassis tubing
- F2.** With occupant, pull the chair backwards.

In stationary position:

- F3.** Always lock as many castors as possible when the chair is to be used in a stationary position.
- F4.** Out-turned castors improve the chair's stability.

Transferring:

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as risky. Visit www.etac.com for manual transfer aids.

F3-F4. It is important to ensure that the castors are locked and pointing outward before carrying out transfers to and from Swift Mobile.

- Fold up the armrest on the transfer side of the chair.
- F5-F7.** Carry out the transfer.

English

Adjustment See Figure G

Height
G1-G7. The height is easily adjusted by inserting the bolt in the required position. See distances in table

Check that both sides have been adjusted to the same height as otherwise stability can be reduced. **G8.** Do not use the chair if the bolt (orange) is not locked.

Swift Mobile	Swift Mobile 24"	Swift Mobile Tilt
--------------	------------------	-------------------

<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
--------------------------	--------------------------	--------------------------

Tilt angle
G9. Fit the gas piston in position A to tilt the seat at angles from -5° to 30°
G10. Fit the gas piston in position B to tilt the seat at angles from 0° to 35°

<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
--------------------------	--------------------------	-------------------------------------

User brake
G11. When adjusting the brake, the brake pin should be set 18 to 22 mm from the rear wheel for good grip.

<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>
--------------------------	-------------------------------------	--------------------------

Legrest See Figure H

H1. The assistant can easily raise and swivel the legrests to the side for the user to sit down in or get up out of the chair. The footrests are detachable.

H2. Various height adjustment options are available by moving the locking pin. The legrests have 6 different height settings.

<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
-------------------------------------	-------------------------------------	-------------------------------------

Tilt function See Figure I

I1. The assistant can adjust the sitting angle with ease by holding in the handle and tilting to the required position. Make sure that the wheels are locked if the tilt function is used when the user is sitting in the chair.

<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
--------------------------	--------------------------	-------------------------------------

Accessories See Figure J

User brake
G11 When adjusting the brake, the brake pin should be set 18 to 22 mm from the rear wheel for good grip.

<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
--------------------------	--------------------------	--------------------------

See Figure J	Sw M.	Sw M. 24"	Sw M. Tilt
J1. Leg and calf rest, angle-variable	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J2. Footrest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J3. Commode pan	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J4. Pan-holder attachments	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J5. Attachments for low pan-holder mounting	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J6. Commode pan holders	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J7. Bucket holders	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J8. Splash guard	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J9. Comfort seat	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J10. Seat cover	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J11. Anti-tip stabilisers including bracket	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J12. Rear wheel kit	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>
J13. Anti-tip stabilisers excluding bracket	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J14. Headrest and bracket	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
J15. Crossbar 605 mm	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J15. Crossbar 665 mm	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J16. Heel straps	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>
J17. Chest belt	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J18. Positioning belt	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

See Figure J	Sw M.	Sw M. 24"	Sw M. Tilt
J19. Hemiplegic armrest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J20. Armrest lock	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J21. Angle adj. footrest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J22. Soft back pad	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J23. Tilter	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J24. Widening kit	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J25. Soft Comfort seat	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J26. Soft Comfort seat - narrow opening	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J27. Comfort padding	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J28. Trunk support	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J29. Armrest cushion, pair	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J30. ROHO pressure-relieving cushion	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J31. Pan	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J32. Lid with handle	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J33. Amputee legrest	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J34. Soft cushion - standard opening	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J35. Soft cushion - oval opening	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
J36. Motor kit	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB