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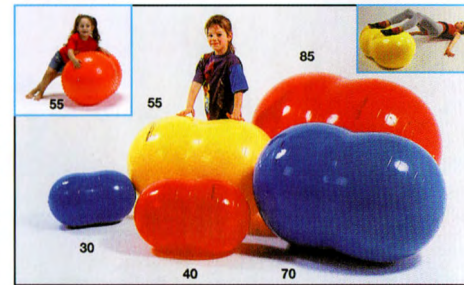
- ❑ The first time you pump up the Roll proceed slowly, especially if you use an electric air compressor. It is better to have the Roll at room temperature when you inflate it.
- ❑ Keep the Rolls away from sources of heat such as sun, lamps, heating ventilators.
- ❑ Be careful to check your clothes and the area where you are using the Rolls for sharp or angular objects that could puncture it. Remember, there is a high risk for punctures when the Roll is used outdoors.
- ❑ In case of puncture, we suggest **not to repair the Roll**. Once repaired it would not be in compliance with safety rules anymore.
- ❑ To clean the Roll wash it with soapy water. Do not use brushes, abrasive sponges or chemical cleaners.
- ❑ The Roll color will fade after long exposure to the sun.
- ❑ Manufacturer and importer assume no liability for any injuries by improper use of the Roll or by non-observance of the instructions.
- ❑ Not suitable for children under 3 years. It contains small parts.
- ❑ For any needs take note of the manufacturer's address.
- ❑ For physiotherapy exercises follow the instructions given by your doctor/therapist.

PHYSIO - ROLL

PAT N° 1766015

(marchio depositato)

LATEX (NRL) FREE

COD. INT. 1990/2004

PHYSIO-ROLL
Ø cm. 30 colour: blue

PHYSIO-ROLL
Ø cm. 40 colour: red

PHYSIO-ROLL
Ø cm. 55 colour: yellow

MESSAGE PEANUT-BALL
Ø cm. 55 colour: orange

PHYSIO-ROLL
Ø cm. 70 colour: blue

PHYSIO-ROLL
Ø cm. 85 colour: red

⚠ WARNING:
CHOKING HAZARD - small parts.
Not for Children under 3 years.

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