# CleanSpace™

wearable protective masks

[www.cleanspacetechnology.com](http://www.cleanspacetechnology.com)

## USER INSTRUCTIONS

■ **CleanSpace EX** PAF-0060

■ **CleanSpace Ultra** PAF-0070

## Contents - Languages

English.....	2 - 31
Danish.....	32 - 63
Estonia.....	64 - 95
Finnish.....	96 - 128
French.....	129 - 162
German.....	163 - 197
Italian.....	198 - 234
Latvian.....	235 - 267
Lithuanian.....	268 - 301
Norwegian.....	302 - 333
Polish.....	334 - 367
Romania.....	368 - 401
Spanish.....	402 - 435
Swedish.....	436 - 466

**PAFtec Australia Pty Ltd**

ABN 24 146 453 554

GF, 16-18 Carlotta St Artarmon NSW 2064 Australia

| T: +61 2 8436 4000 | E: EU.sales@paftec.com | E: sales@paftec.com

## 1. Contents

1.	Contents .....	2
2.	Foreword.....	2
3.	List of Warnings within these User Instructions .....	3
4.	Special or Critical User Instructions .....	3
5.	Limitations of Use .....	4
6.	System Components .....	5
7.	Controls and Indicators .....	7
8.	Using the CleanSpace respirator with Half Masks .....	8
9.	Using the CleanSpace Respirator with Full Face Masks .....	16
10.	Entering and Exiting the Contaminated Area .....	21
11.	Working in Noisy Environments .....	22
12.	Changing the Filter .....	22
13.	Battery Information .....	25
14.	Cleaning.....	27
15.	Recalibration.....	28
16.	Approvals.....	28
17.	Specifications .....	30
18.	Product Warranty.....	31

**Important:** Before use, the wearer must read and understand these user instructions. Keep these user instructions for reference.

Read these instructions in conjunction with the appropriate CleanSpace filters and accessories instructions.

### **WARNING**

*This product is part of a system that helps protect against certain airborne contaminants. Misuse may result in sickness or death. For proper use, see an Occupational Health Specialist, these user instructions or contact PAFtec Customer Service on +612 8436 4000*

## 2. Foreword

Read all instructions and warnings before using this device. Keep these user instructions for reference. If you have questions regarding this system contact CleanSpace Customer Service on +612 8436 4000 or e-mail [support@paftec.com](mailto:support@paftec.com).

### 3. List of Warnings within these User Instructions

This product is part of a system that helps protect against certain airborne contaminants.

**WARNING:** Properly selected, used and maintained respirators help to protect against certain airborne contaminants by reducing concentrations in the wearer's breathing zone. It is essential to follow all instructions and government regulations on the use of this product, including wearing the complete respirator system during all times of exposure, in order for the product to help protect the wearer. Misuse of respirators may result in overexposure to contaminants and lead to sickness or death. For proper use, see an Occupational Health Specialist, these user instructions or contact CleanSpace Customer Service on +612 8436 4000

No PAPR can fully protect you if you are not clean shaven. CleanSpace Respirator is not suitable for users with facial hair.

**IF A SATISFACTORY FIT FACTOR CANNOT BE ACHIEVED WITH ANY OF THE MASKS, CLEANSPACE PAPRS MUST NOT BE USED**

CleanSpace PAPRS are a complete respiratory protection system. In addition some combinations of components have been approved for use in some explosive atmosphere. Refer to Section 16. Product and Accessory Information.

Not all of the components shown below have been approved for use in explosive atmospheres. See Approvals Section below for details.

The IECEx / ATEX approvals apply only when all components are IECEx / ATEX approved. Therefore when purchasing spare parts and accessories, always make sure of their proper approval if the equipment is to be used in an explosive atmosphere. Not all components and accessories shown in this manual carry IECEx / ATEX approval.

If working in intrinsically safe environments, before each use with the intrinsically safe masks - check that the black conductive coating on the mask has not been significantly damaged.

The battery shall only be charged in non-Hazardous areas. Move out of the hazardous area before charging CleanSpace PAPRS

**WARNING:** For CleanSpace EX - Charging voltage  $U_m = 14.7V$  from an / via an EX certified intrinsically safe Category I (Ex ia) device

CleanSpace EX uses a different charger from all other CleanSpace devices. Do not attempt to charge your CleanSpace EX with a charger from a CleanSpace2 or CleanSpace ULTRA. The EX Charger is also not suitable for charging those devices.

**WARNING:** Be sure to remove the Seal Check Cap before entering the contaminated area. The Cap blocks the exhalation valve, making it more difficult for your exhaled air to be expelled from the mask. Failure to remove the cap will lead to build-up of Carbon Dioxide in the mask, and may result in headache or dizziness. Never leave the Cap in place for more than 2 minutes!

**DO NOT REMOVE THE RESPIRATOR UNTIL YOU HAVE VACATED THE CONTAMINATED AREA** unless you have pressing health reasons to do so (for instance you are experiencing dizziness and believe removing the respirator while you leave the contaminated area may help)  
IT IS ESSENTIAL THAT THE CORRECT FILTER TYPE IS SELECTED FOR THE CHOSEN APPLICATION.

If the blocked filter alarm is triggered (2 beeps, repeated every second, red LED flashes), leave the contaminated area immediately and replace the filter.

Operating the respirator after the blocked filter alarm has sounded can cause the flow to fall below the manufacturer's minimum designed flow, which may result in overexposure to contaminants and lead to sickness or death.

Use only CleanSpace filters with the CleanSpace PAPRS power units. Use of other filters may result in overexposure to contaminants and lead to sickness or death.

Do not charge the on-board battery with unapproved chargers, in enclosed cabinets without ventilation, near flammable liquids or gasses, or near sources of high heat

Do not immerse the device in water

Do not use, charge or store the device outside the recommended temperature limits

If the battery alarm sounds (3 beeps, repeated every second), leave the contaminated area immediately and re-charge the battery.

Operating the respirator after the low battery alarm has sounded can cause the flow to fall below the manufacturer's minimum designed flow, which may result in overexposure to contaminants and lead to sickness or death.

**IN THE EXTREMELY RARE CIRCUMSTANCE THAT THE BATTERY IS DAMAGED AND ELECTROLYTE COMES IN CONTACT WITH EYES, FLUSH WITH WATER IMMEDIATELY AND SEEK URGENT MEDICAL ATTENTION**

**WARNING: NEVER DRY THE MASK OR EXHALATION VALVE WITH A CLOTH THAT MAY LEAVE BEHIND LINT.**

**Lint contamination of the exhalation valve may cause it to leak, resulting in overexposure to contaminants and lead to sickness or death.**

**WARNING:** The blower contains a battery, sensitive electronics and a motor. NEVER immerse it in water or use anything wetter than a damp cloth to clean it.

**WARNING: NEVER CLEAN THE FILTER WITH COMPRESSED AIR OR A BRUSH. HEPA FILTERS ARE EASILY DAMAGED BY THE USE OF COMPRESSED AIR OR BY BRUSHING.**

**Misuse of CleanSpace Respirator by cleaning the filter may result in overexposure to contaminants and lead to sickness or death.**

You must recalibrate the internal pressure sensor any time that the CleanSpace PAPR is exposed to changes in temperature of more than 20°C.

### 4. Special or Critical User Instructions

CleanSpace PAPR contains a system that detects and corrects for changes in altitude. This system requires re-calibration whenever it experiences a change in temperature of more than 20°C. See Section 15 for instructions on how to recalibrate CleanSpace PAPR. **If using a CleanSpace EX (PAF-0060) - The IECEx / ATEX approvals apply only when all components are IECEx / ATEX approved. Therefore when purchasing spare parts and accessories, always make sure of their proper approval if the equipment is to be used in an explosive atmosphere. Not all components and accessories shown in this manual carry IECEx or ATEX approval. See Approvals (below) for details. The battery must not be charged whilst the respirator is located in an explosive atmosphere.**

## 5. Limitations of Use

Use this respirator strictly in accordance with all instructions in these user instructions. Never modify or alter this product.

- **DO NOT USE** for respiratory protection against unknown atmospheric contaminants or when concentrations of contaminants are unknown or **immediately dangerous to life or health** (IDLH).
- **Do not use if the oxygen concentration is below 19.5%**
- **Do not remove the respirator until you have left the contaminated area**, unless you have pressing health reasons to do so (for instance you are experiencing dizziness and believe removing the respirator while you leave the contaminated area may help).
- Only suitable for use by clean-shaven personnel. Facial hair under the seal reduces protection and is not permitted.
- Unless used with a CleanSpace Full Face Mask, this respirator DOES NOT PROTECT YOUR EYES. In conditions that may damage or irritate eyes, use protective eyewear.
- Only use CleanSpace-approved filters and accessories
- Do not use the respirator unless it is powered and running normally.
- Do not use the respirator while it is being charged via the AC adaptor.
- Do not use in airborne contaminant concentrations above those specified in your national regulations
- Only for use by trained personnel.
- Filters need to be regularly changed. The frequency of change depends on use and the concentration of contaminants in the atmosphere.
- Do not use for escape purposes. National regulations may impose specific limitations on the use of filters depending on the filter class and the facemask used.

### LEAVE THE CONTAMINATED AREA IMMEDIATELY IF:

- Respirator warning lights or sounds activate for low battery or blocked filter
- Any part of the respirator is damaged
- Air flow into the mask decreases or stops
- Breathing becomes difficult or increased resistance occurs
- You feel dizzy or your airway is irritated
- You can taste or smell contaminants

Your CleanSpace respirator is suitable for use in the following atmospheric conditions:

- Temperature: -10°C to 45°C
- Relative humidity: 0 to 90% non-condensing

The CleanSpace Respirator will stop functioning if its internal temperature rises above 60°C (ambient 50°C) or falls below -10°C

If the respirator has been used in an area that has caused it to become contaminated with a substance requiring special decontamination procedures it should be placed in a suitable container and sealed until it can be decontaminated.

Do not disassemble the CleanSpace respirator case. There are no user serviceable parts inside.

Failure to follow all instructions on the use of this product, and/or failure to use the CleanSpace respirator during times of exposure, may lead to adverse effects on the wearer's health and may render the warranty void.

### WARNING


*This product is part of a system that helps protect against certain airborne contaminants. Misuse may result in sickness or death. For proper use, see an Occupational Health Specialist, these user instructions or contact PAFtec Customer Service on +612 8436 4000*

## 6. System Components

*CleanSpace Respirator is a complete respiratory protection system. In addition some combinations of components have been approved for use in some explosive atmospheres. Not all of the components shown below have been approved for use in explosive atmospheres. See Approvals below for details.*

1A PAF-0064	Half Mask – Small, EX	2A PAF-0035	Filter P3 TM3 P SL R Standard	4A PAF-0073	Head Harness- Half mask
1B PAF-0062	Half Mask – Medium, EX	2B PAF-0037	Filter P3 TM3 P SL R High Capacity	4B PAF-1018	Head Harness – Full Face
1C PAF-0063	Half Mask – Large, EX	2C PAF-0048	Combined filter A1P3 (AU)	5 PAF-0034, 60, 70	Power Unit 2, EX & Ultra
1D PAF-0033	Half Mask – Small	2D PAF-0047	Combined filter ABE1P3 (AU)	6A PAF-1012	Neck support (small)
1E PAF-1010	Half Mask – Medium	2E PAF-0050	Combine filter A1TM3 P SL R (EU)	6B PAF-1013	Neck Support (Large)
1F PAF-0027	Half Mask – Large	2F PAF-0051	Combine filter ABE1TM3 P SL R (EU)	7A PAF-1005	Flow Test Cap
1G pAF-1014	Full Face Mask	3A PAF-0042, 65 & 72	Filter Cover 2, EX & Ultra	7B PAF-0040, 68	Battery Chargers 2, Ultra & EX
		3B PAF-0038	Filter Adaptor (large case filters)	7C PAF-1009	Seal Check cap

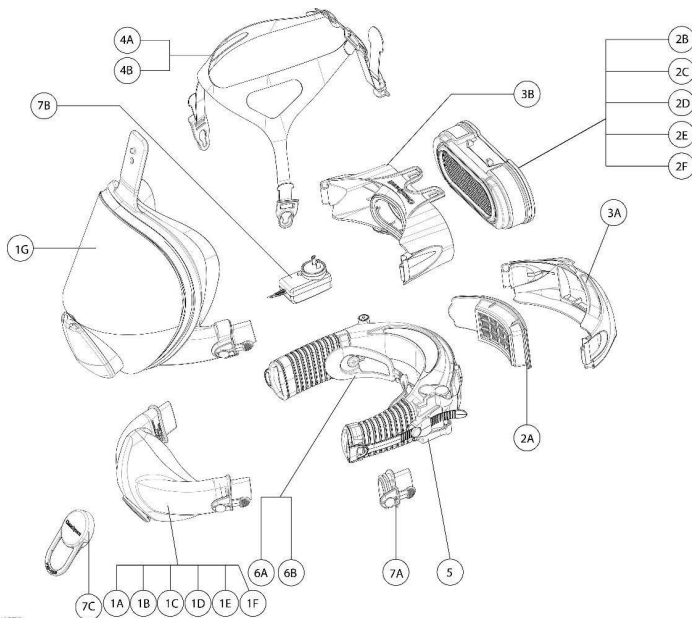
Figure 1 - System Components

XPREV1

## PARTS OF THE RESPIRATOR

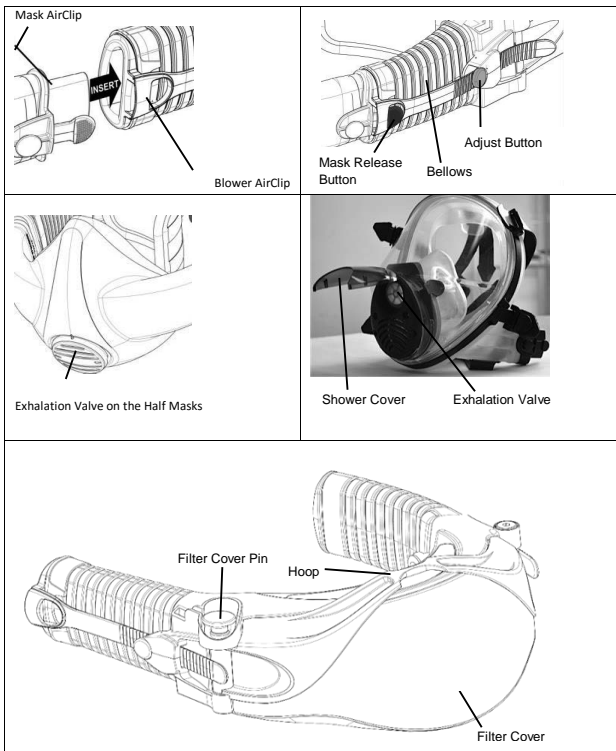Through this manual reference is made to various commonly-used components and features of the machine. Familiarise yourself with these parts before reading the rest of the manual. See Figure - Parts of the respirator.

## 7. Controls and Indicators

### POWER BUTTON

This button is used to switch between the three (3) operating Modes: "On", "Standby" and "Off".

- i) **Standby Mode** is when the green battery indicator lights are on, the motor is not running and there is no airflow to the mask. The respirator will automatically switch to Standby Mode within ten (10) seconds of the user taking off the respirator. If the CleanSpace Respirator is in Off Mode it will switch to Standby Mode when the Power button is pressed.
  - ii) **On Mode** is when the respirator is being worn, the motor is running and there is air flow to the mask. The motor will start (called On Mode) when the respirator detects a change in pressure in the mask triggered by your breathing. In On Mode you should hear the motor running and feel the airflow on your face. You can also switch to On Mode (start the motor) from Standby Mode by pressing the Power button once.
  - iii) **Off Mode** is when the green battery indicator lights are off and the motor is not running and there is no air flow to the mask. The respirator automatically switches into Off Mode three (3) minutes after the respirator has been removed from the user's face and the sensors detect that there is no breathing.
- To conserve battery life, the CleanSpace Respirator is designed to automatically switch from On Mode to Standby then to Off Mode when not being worn.

Important: CleanSpace Respirator switches into On Mode when the respirator is in Standby Mode and the wearer starts to breathe. The respirator can only switch into On Mode from Standby Mode.

### FLOW TEST BUTTON

This button is used to check that the respirator is able to deliver its minimum designed flow. Pressing it once, when the CleanSpace Respirator is in Standby Mode (not ON Mode), starts the flow test which lasts about 5 seconds. See Section 9 for instructions on running the flow test.

### BATTERY INDICATOR LIGHTS

CleanSpace Respirator is equipped with an indicator of battery charge. There are three (3) battery indicator lights. With the charger plugged in and three green lights lit, the battery is fully charged and has minimum four (4) hours of operating time. The respirator should be fully charged before use.

CleanSpace Respirator will function with one (1) or two (2) green battery lights illuminated. If two (2) battery indicator lights are on, the respirator has between 25% and 75% charge and will typically operate for between 1 and 3 hours. If only one (1) battery indicator light is on, the battery has less than 20% charge and we recommend that you charge the respirator before use.

When the battery approaches a level at which it would not be able to supply the Manufacturer's Minimum Design Flow, an alarm sounds (3 beeps, repeated once per second). All green battery lights are extinguished. **If the low battery alarm sounds you must leave the contaminated area immediately and recharge the battery.**

*Operating time is strongly affected by work rate, altitude, and other factors. The operating times quoted above are average durations at moderate work rates at sea level. Actual operating times may vary widely from average durations.*

### FILTER WARNING ALARM

CleanSpace Respirator has a Filter Warning Alarm, which is triggered when the filter is blocked. **If the Filter Warning Alarm sounds** (two beeps, repeated once per second) or the Filter Warning Alarm light comes on, **you must move out of the contaminated area, and change the filter.**

## 8. Using the CleanSpace respirator with Half Masks

### MASK FITTING –HALF MASKS

If you are using one of the following masks, read this section to learn how to fit:

Intrinsically Safe approved	NON – Intrinsically Safe
PAF-0064 Mask Half Small – Intrinsically Safe approved	PAF-0033 Mask Half Small
PAF-0062 Mask Half Medium - Intrinsically Safe approved	PAF-1010 Mask Half Medium
PAF-0063 Mask Half Large - Intrinsically Safe approved	PAF-0027 Mask Half Large

Before you use CleanSpace Respirator you must determine the right mask for your face and know how to adjust the machine to achieve a good fit.

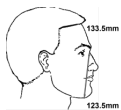
It is vital that your mask is the right size for your face and fits properly. **Mask fitting must be carried out by a specialist / designated mask fitter.** The fit must be confirmed by a **quantitative fit test**.

To carry out a quantitative fit test your specialist mask fitter will require **PAF-0025 PortaCount Adaptor** and the accompanying **S005-7174 Quantitative Fit Test Instructions**. The PortaCount Adaptor is an accessory that fits between the mask and the exhalation valve and allows the air in the mask to be sampled. It comes with detailed instructions on how to carry out a quantitative fit test.

**No PAPR can fully protect you if you are not clean shaven. CleanSpace Respirator is not suitable for users with facial hair.**

### MASK SELECTION GUIDELINES

No set of guidelines can ensure that you have the right size mask for your face. You must confirm the fit with a quantitative fit test. But the table below will help your mask fitting specialist to select the mask most likely to fit you, and may therefore save time.

FACE LENGTH	LIP LENGTH		
	34.5mm	43.5mm	61.5mm
133.5mm	First Try: <b>MEDIUM*</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM*</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>
123.5mm	First Try: <b>MEDIUM</b> Then Try: <b>SMALL*</b>	First Try: <b>MEDIUM</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>
113.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>
103.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>
93.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>

\* On narrow faces, the action of pulling on the arms of the mask to tighten it causes the height of the mask to increase. The narrower the face, the more likely a smaller mask will be a better fit.

Once you have achieved an acceptable mask fit and confirmed it with a quantitative fit test, record your information in the table below.

Name	
Mask Size	
Neck Support	Small / Large (Circle one)
Date of Quantitative Fit Test	
Fit Factor on Test	

**IF A SATISFACTORY FIT FACTOR CANNOT BE ACHIEVED WITH ANY OF THE MASKS, CLEANSPACE RESPIRATOR MUST NOT BE USED**

## USING CLEANSPACE RESPIRATOR WITH THE HALF MASK.

Complete the following steps each time you use CleanSpace Respirator

### STEP 1 – INSPECT THE EQUIPMENT

Before each entry into a contaminated area, the following inspections must be performed:

- Visually check the entire CleanSpace Respirator system including the blower, mask, harness, filter adaptor and filter. If parts are missing or damaged replace them only with approved CleanSpace Respirator parts before proceeding. Check the top and bottom case of the blower, and the Filter Adaptor (if fitted) for cracks or other damage. Do not use the device if there is any damage.
- Remove the filter and check it carefully. The seal must be clean and free from damage of any kind. If necessary it can be cleaned with a cloth dampened with water. Examine the surfaces behind the filter carefully for any sign that dust has leaked past a damaged seal. If found, replace the filter. The body of the filter must not be cracked or show any sign of damage. Examine the filter carefully for any sign that it has sustained an impact or been scratched. If any sign of impact or scratching is found, discard the filter. At the end of your inspection, refit a filter and check that it is secure (see Changing the Filter, below)

**WARNING: NEVER CLEAN THE FILTER WITH COMPRESSED AIR OR A BRUSH. HEPA FILTERS ARE EASILY DAMAGED BY THE USE OF COMPRESSED AIR OR BY BRUSHING.**

*Misuse of CleanSpace Respirator by cleaning the filter may result in overexposure to contaminants and lead to sickness or death.*

- Check that the battery is fully charged by pressing the power button. All three battery LEDs must light. Check that the battery has enough charge to complete the work period.
- Check both bellows for splits or holes. Check that the bellows have not become distorted so as to partially or fully close the air path to the mask.
- Check the mask to ensure that there are no cracks, tears or dirt; check the mask is not distorted.
- Check the exhalation valve for damage or dirt build up. If it is dirty, remove the exhalation valve cover. Remove any dirt, hairs or anything that could affect the seal of the valve against its seat. Check that the valve seat is clean. Reinstall the valve cover. If the valve is damaged, replace the mask.
- Check the harness is intact. It must be adjusted to support some of the weight of the CleanSpace Respirator.

### STEP 2 – CALIBRATE THE POWER UNIT

CleanSpace Respirator contains a system that detects and corrects for changes in altitude. **This system requires re-calibration whenever it experiences a change in temperature of more than 20°C. To re-calibrate:**

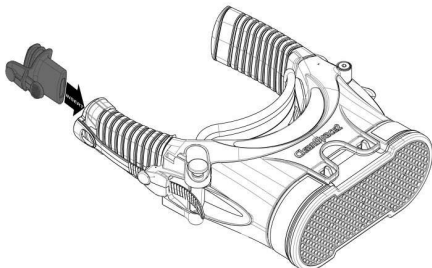
1. With CleanSpace Respirator in standby mode (one or more green LEDs lit), press and hold **both** the Power button and the FLOW TEST button. *If no LEDs are lit, press the Power button once to enter standby mode.*
2. When both the blue and red LEDs light, release both buttons.
3. Hold the CleanSpace Respirator still until the red and blue LEDs turn off (10 – 15 seconds)
4. Calibration is complete.

CleanSpace Respirator also recalibrates its pressure sensor any time it is left **undisturbed in Standby Mode** for more than 20 seconds.

### STEP 3 – TEST FLOW RATE ON THE POWER UNIT

*This test checks that the machine is able to deliver the Manufacturer's Minimum Design Flow of 120 litres/minute.*

1. Fit the yellow Flow Test Cap to the **left** bellow. Note: The respirator shown here with Filter Adaptor fitted, but test is the same regardless of respirator configuration.

To avoid unnecessary filter changes, it is important to fit the Flow Test Cap to the **LEFT** bellow.

No air flows out of the right bellow. Fitting the Flow Test Cap to the right bellow will result a **FAIL** result, even if the filter is clean and battery fully charged.

2. Make sure nothing is blocking the Flow Test Cap or the filter inlet. Place CleanSpace Respirator flat on a table or other support.
3. Press and release the button marked "Flow Test"
4. CleanSpace Respirator automatically runs the Flow Test. The motor will run fast and air will be discharged from the Flow Test Cap.
5. After 2 seconds CleanSpace Respirator reports the result of the test using the LEDs on the keypad. Use the table below to interpret the LEDs.

LIGHTS	Meaning
<b>3 LEDs:</b>	<b>PASS (Excellent: flow &gt;180 l/min)</b>
<b>2 LEDs:</b>	<b>PASS (Good)</b>
<b>1 LED:</b>	<b>PASS (Acceptable)</b>
<b>ALL LEDs</b>	<b>FAIL (Flow &lt;120 l/min)</b>
<b>FLASH</b>	<p><b>Do not use CleanSpace Respirator until a new filter has been fitted and / or battery charged and the test has been repeated with a PASS result.</b></p> <p><i>Reset the respirator by pressing the Power button. Fully charge the battery and / or replace the filter. Repeat the flow test. If filter is new and battery fully charged but CleanSpace Respirator fails the test, contact PAFtec and do not use until it has been evaluated</i></p>

6. Remove the yellow Flow Test Cap and store for later use.

### STEP 4 - DON THE RESPIRATOR AND HALF MASK

**NOTE:** Please read **STEP 4 all the way through** before beginning to don CleanSpace PAPP

Locate your blower, mask, neck support and harness. **Make sure the mask and neck support are of the size that you used for your last successful quantitative fit test.** These sizes should be recorded in this manual (see Section 8 above).

### Fit a Neck Supports to CleanSpace PAPR

CleanSpace Respirator comes supplied with two neck supports:

- PAF-1012 NECK SUPPORT SMALL
- PAF-1013 NECK SUPPORT LARGE

Select a neck support. PAF-1012 is more suited to smaller faces while PAF-1013 fits people with larger faces.

The neck support has keyhole openings at each end which snap over buttons on the respirator case. Place the neck support in position against the buttons and then press firmly backwards (towards the respirator) to engage the buttons in the keyholes.

### Fit a mask to CleanSpace PAPR

- Examine the mask AirClips. One is open and one is closed. Find the one that is **open**. Seen from above it is on the left hand side.
- Locate the left hand blower AirClip. It is on the same side of the blower as the "POWER" button.
- Figure shows the Left hand (OPEN) mask AirClip

Join the AirClips, so that the mask is attached to the blower. See Figures below


Figure A. Join AirClip

Figure B. Mask on Blower. Note: connection is on left hand side

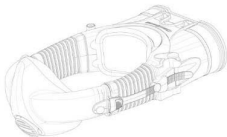**IMPORTANT:** Check that the mask is the right way up. The pointed, nose section of the mask must face up, in the same direction as the keypad. See Figure. Leave the other mask arm and bellow free until you are fitting the respirator.

**Fit the Harness to the mask**

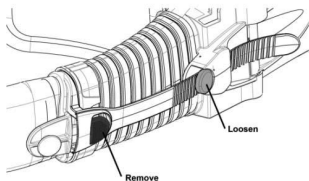
*Orient the harness so there is a clip with a keyhole next to each arm of the mask and the rear harness strap is close to the respirator.*


*Clip the harness to the mask.*

**Familiarise yourself with the Adjust Buttons and the Mask Release Buttons.**

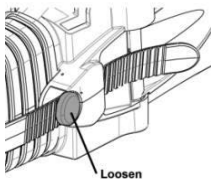Mask Release Button and Adjust Button


**Loosen both bellows to their widest opening**

*Press the adjust button on the blower and pull on the bellows to loosen and extend the bellows.*

<p><b>Place the machine in Standby Mode.</b> Press the Power button once. The GREEN lights of the Battery Indicator should come on.</p>	
<p><b>Don the respirator and start to breathe.</b> Place the blower behind your neck with the bellows and mask hanging down one side.</p>	
<p>Join the right hand Mask AirClip to the right hand respirator AirClip</p>	
<p>Find the Harness rear strap and the harness rear clip on the end of it. Holding the rear strap pull it back over your head.</p>	
<p>Locate the hook on the bottom end of the strap. Snap the hook onto the hoop on the filter cover of CleanSpace Respirator so that the rear strap supports the weight of the respirator.</p>	

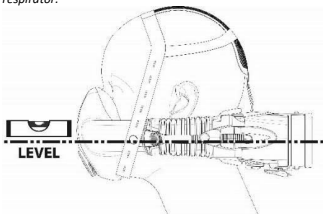**Adjust the height of the respirator** *The Harness should take a little of the weight of the mask and support the blower so that the CleanSpace Respirator system sits level on your head. If necessary adjust the rear strap to raise or lower the respirator.*

**Tighten the mask on your face.** Tighten the mask until you do not feel any leaks between the mask cushion and your face.

*To tighten the mask, place one hand against the back of the blower and the other over the front of the mask. Push the blower forwards and the mask back, tightening the fit. You will hear a series of clicks as the mask is tightened. Be careful to tighten each side by the same amount*

*While adjusting the mask fit, watch for these signs that the mask is not fitted correctly*

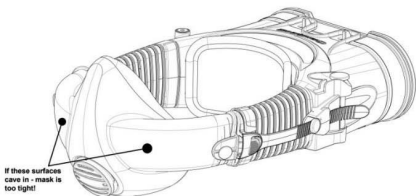
**No Gaps:** *There should be no visible gaps between the user's face and the mask.*

**No Creases:** *If the mask is pulled too tight, or is too large for the face, creases may develop at around the level of the mouth or on the centreline at the bottom of the cushion. Loosen the mask a little or try a smaller mask.*

**Look Down:** *Have the user look down while you check for a gap between the bridge of the nose and the mask. Also check that the mask has not come loose around the chin. Try to fit the chin further into the mask. If that does not work, try a smaller mask.*


**No air in the eyes:** *With CleanSpace running, have the user check that there is no leak into their eyes. If there is, try tightening the mask, or loosen the harness a little (dropping the height of CleanSpace on the face). If those adjustments do not fix the problem, try a different size mask.*

**Check for Distortion:** *You should be able to achieve a good fit without the mask being uncomfortably tight. **Beyond a certain point, tightening the mask makes the fit worse not better.** Check the outer surfaces of the air tubes. They should curve out towards you. If they are curved in towards the face, you have the mask on too tight.*

## STEP 6 – CHECK MASK SEAL

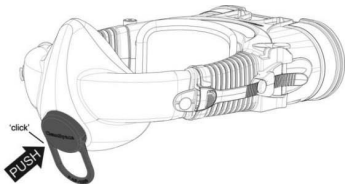*It is essential to do a fit check every time you wear a CleanSpace Respirator.* Locate PAF-1009, the red Seal Check Cap. Fit Seal Check Cap over the exhalation valve in the mask. The Cap should snap into place. See figure below.

- Check that no air is flowing from the exhalation valve. Breathe normally.
- Using your fingers, feel around the perimeter of the mask for leaks. You will feel any leak as a cool flow of air over your finger. For greater sensitivity breathe out firmly to raise the mask pressure. Wetting your fingers will make it easier to feel tiny leaks.
- If necessary tighten the mask. To tighten the mask, place one hand against the back of the blower and the other over the front of the mask. Push the blower forwards and the mask back, tightening the fit. You will hear a series of clicks as the mask is tightened.
- Be careful to tighten each side by the same amount.

**Do not over** tighten the mask! If the front of the mask starts to become concave, it is too tight and may leak. Press the Adjust Buttons to loosen the mask a little.


After each tightening, feel for leaks again

Tilt your head down (look at the ground) and up (look at the sky). Check that there are still no leaks. Look right and left, checking for leaks. Adjust if necessary.

Once you can feel no leaks from the mask, the seal check is complete.

**IF A SATISFACTORY FIT CANNOT BE ACHIEVED, DO NOT ENTER THE CONTAMINATED ZONE**

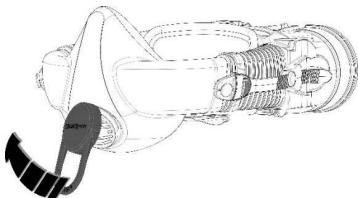Remove the Seal Check Cap by gently **twisting** the handle so that one side of the cap comes loose from the exhalation valve. Be careful not to dislodge the exhalation valve cover. See Figure on the following page.

**Be sure to remove the Seal Check Cap before entering the contaminated area. The Cap blocks the exhalation valve, making it more difficult for your exhaled air to be expelled from the mask. Failure to remove the cap will lead to build-up of Carbon Dioxide in the mask, and may result in headache or dizziness.  
Never leave the Cap in place for more than 2 minutes!**

## 9. Using the CleanSpace Respirator with Full Face Masks

### MASK FITTING –FULL FACE MASKS

If you are using PAF-1014 Full Face Mask, read this section to learn how to fit it.

**PAF-1014 is a Medium / Large mask. It may not be appropriate for you if your face is smaller than average or your hairline crosses the seal. Do not use the mask unless the fit has been confirmed by a quantitative fit test.**

Before you use any mask it is vital that you ensure it fits properly. **Mask fitting must be carried out by a specialist / designated mask fitter.** The fit must be confirmed by a **quantitative fit test**.

To carry out a quantitative fit test your specialist mask fitter will require **PAF-1015 Full Face Mask Adaptor for Fit Testing** and the accompanying **5005-7184 Full Face Quantitative Fit Test Instructions**. The Mask Adaptor is an accessory that fits between the mask and the exhalation valve and allows the air in the mask to be sampled. It comes with detailed instructions on how to carry out a quantitative fit test.

***The Standards state that No tight fitted respirator can fully protect you if you are not clean shaven.***

Once you have achieved an acceptable mask fit and confirmed it with a quantitative fit test, record your information in the table below.

Name	
Mask Type	Full Face Mask PAF-1014
Neck Support	Small / Large (Circle one)
Date of Quantitative Fit Test	
Fit Factor on Test	

**IF A SATISFACTORY FIT FACTOR CANNOT BE ACHIEVED WITH ANY OF THE MASKS, CLEANSPACE RESPIRATOR MUST NOT BE USED**

### DONNING THE CLEANSPACE RESPIRATOR WITH THE FULL FACE MASK

Complete the following five steps each time you use CleanSpace Respirator with an FULL FACE MASK.

## STEP 1 – INSPECT THE EQUIPMENT

Before each entry into a contaminated area, the following inspections must be performed:

- Visually check the entire CleanSpace Respirator system including the blower, mask, harness, filter adaptor and filter. If parts are missing or damaged replace them only with approved CleanSpace Respirator parts before proceeding. Check the top and bottom case of the blower, and the Filter Cover or Filter Adaptor (if fitted) for cracks or other damage. Do not use the device if there is any damage.
- Remove the filter and check it carefully. The seal must be clean and free from damage of any kind. If necessary it can be cleaned with a cloth dampened with water. Examine the surfaces behind the filter carefully for any sign that dust has leaked past a damaged seal. If found, replace the filter. The body of the filter must not be cracked or show any sign of damage. Examine the filter carefully for any sign that it has sustained an impact or been scratched. If any sign of impact or scratching is found, discard the filter. At the end of your inspection, refit a filter and check that it is secure (see Changing the Filter, below)

**WARNING: NEVER CLEAN THE FILTER WITH COMPRESSED AIR OR A BRUSH. HEPA FILTERS ARE EASILY DAMAGED BY THE USE OF COMPRESSED AIR OR BY BRUSHING.**

*Misuse of CleanSpace Respirator by cleaning the filter may result in overexposure to contaminants and lead to sickness or death.*

- Check that the battery is fully charged by pressing the power button. All three battery LEDs must light. Check that the battery has enough charge to complete the work period.
- Check both bellows for splits or holes. Check that the bellows have not become distorted so as to partially or fully close the air path to the mask.
- Check the mask to ensure that there are no cracks, tears or dirt; check the mask is not distorted.
- Check the exhalation valve for damage or dirt build up. If it is dirty, flip up the mask rain cover. Remove any dirt, hairs or anything that could affect the seal of the valve against its seat. Check that the valve seat is clean. Reinstall the valve cover. If the valve is damaged, replace the mask.
- Check the harness is intact.

## STEP 2 – CALIBRATE THE POWER UNIT

CleanSpace Respirator contains a system that detects and corrects for changes in altitude. **This system requires re-calibration whenever it experiences a change in temperature of more than 20°C. To re-calibrate:**

- With CleanSpace Respirator in standby mode (one or more green LEDs lit), press and hold **both** the Power button and the FLOW TEST button.
- *If no LEDs are lit, press the Power button once to enter standby mode.*
- When both the blue and red LEDs light, release both buttons.
- Hold the CleanSpace Respirator still until the red and blue LEDs turn off (10 – 15 seconds)
- Calibration is complete.

CleanSpace Respirator also recalibrates its pressure sensor any time it is left **undisturbed in Standby Mode** for more than 20 seconds.

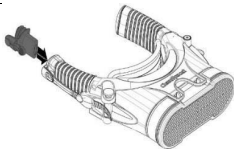
## STEP 3 – TEST FLOW RATE

*This test checks that the machine is able to deliver the Manufacturer's Minimum Design Flow of 120 litres/minute.*

Fit the yellow Flow Test Cap to the left bellow.

**It is important to fit the Flow Test Cap to the LEFT bellow. No air flows out of the right bellow. Fitting the Flow Test Cap to the right bellow will result a FAIL result, even if the filter is clean and battery fully charged.**

Note: respirator shown here with Filter Adaptor fitted, but test is the same regardless of respirator configuration

**STEPS FOR TESTING THE FLOW RATE:**

- Make sure nothing is blocking the Flow Test Cap or the filter inlet. Place CleanSpace Respirator flat on a table or other support.
- Press and release the button marked "Flow Test"
- CleanSpace Respirator automatically runs the Flow Test. The motor will run fast and air will be discharged from the Flow Test Cap.
- After two (2) seconds CleanSpace Respirator reports the result of the test using the LEDs on the keypad. Use the table below to interpret the LEDs.

LIGHTS	Meaning
3 LEDs:	PASS (Excellent: flow >180 l/min)
2 LEDs:	PASS (Good)
1 LED:	PASS (Acceptable)
ALL LEDs FLASH	<p>FAIL (Flow &lt;120 l/min)</p> <p><b>Do not use CleanSpace Respirator until a new filter has been fitted and / or battery charged and the test has been repeated with a PASS result.</b></p> <p><i>Reset the respirator by pressing the Power button. Fully charge the battery and / or replace the filter. Repeat the flow test. If filter is new and battery fully charged but CleanSpace Respirator fails the test, contact PAFtec and do not use until it has been evaluated</i></p>

- Remove the yellow Flow Test Cap and store for later use.

**STEP 4 - DON THE RESPIRATOR WITH FULL FACE MASK**

**NOTE:** Please read STEP 4 all the way through before beginning to don CleanSpace Respirator

Locate your blower, your mask and your neck support.

**Fit a Neck Support**

When CleanSpace Respirator is used with a mask the neck support is for comfort only and does not form part of the seal. Nevertheless you may find that comfort is improved by selecting the right size of neck support as explained below. In particular, if your job involves a lot of vigorous movement, selecting the right size neck support and adjusting the machine correctly will stabilise the respirator against your neck and reduce distracting movement.

**FITTING THE NECK SUPPORT**

CleanSpace Respirator comes supplied with two neck supports:

- PAF-1012 NECK SUPPORT SMALL
- PAF-1013 NECK SUPPORT LARGE

Select a neck support. PAF-1012 is more suited to smaller faces while PAF-1013 fits people with larger faces.

The neck support has keyhole openings at each end which snap over buttons on the respirator case. Place the neck support in position against the buttons and then press firmly backwards (towards the respirator) to engage the buttons in the keyholes.

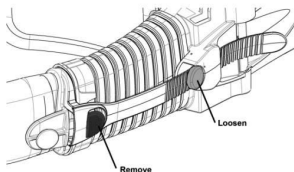
## CleanSpace Respirators


## USER INSTRUCTIONS

**Familiarise yourself with the Adjust Buttons and the Mask Release Buttons.**

Mask Release Button and Adjust Button

**Loosen both bellows to their widest opening**

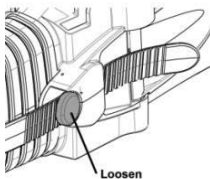Press the adjust button on the blower and pull on the bellows to extend them

**DON THE MASK**

If you have long hair, it is recommended you tie the hair back so that it does not interfere with the seal between the mask and your face.

Loosen all five straps on the mask harness to their fullest extent

Hold the mask in one hand while you use the other hand to pull the harness back and away from the mask

Place your chin in the cup of the mask face seal

And

Pull the harness over your head

Settle your face into the mask face seal. Gently tighten each harness strap in turn, starting with the bottom straps. As you pull each strap, use your other hand to steady the mask on your face.

Continue to adjust the straps, loosening and tightening as necessary, until the mask face seal presses evenly on your face around its entire length.

**CHECK THAT THE MASK SEAL DOES NOT CROSS YOUR HAIRLINE**

Check all the way around the mask seal, paying particular attention to your forehead and temples. **The seal must not cross your hairline.**

**IF YOU CANNOT ADJUST THE MASK TO AVOID YOUR HAIRLINE, THE MASK IS NOT SUITABLE FOR YOU AND MUST NOT BE WORN**

**Check that there is no loose hair between the silicone mask seal and your face**

You will not be able to achieve a good fit if there are any strands of hair between the mask and your face.

**Carry out a negative pressure seal check**

Using your finger or thumb, cover the air inlet (on the left hand mask air clip).

Breathe in sharply. You should not be able to draw any air into the mask. The mask should be sucked in towards your face as you inhale. Listen for squeaking or whistling noises which indicate air is leaking past the seal.

Hold your breath for 10 seconds. The mask should stay collapsed against your face. If the mask seal slowly recovers (mask moves away from your face) there is a leak. Readjust the mask fit and repeat the negative pressure seal check.

**IF A SATISFACTORY FIT CANNOT BE ACHIEVED, DO NOT ENTER THE CONTAMINATED ZONE**

**Place the machine in Standby Mode** Press the Power button once

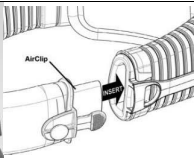
**Don the respirator and start to breathe.**

Place the respirator behind your head, resting on your shoulders. **Note that the hoop on the filter cover must be facing upwards.**

Grasp the left-hand mask AirClip in one hand and the left-hand respirator AirClip in the other. Join them. Pulling down on the mask AirClip to move it away from the harness makes this process easier.

Note Orientate the power unit with the keypad facing up

Breathe normally. CleanSpace Respirator should start.

If the motor does not start, it was not in Standby Mode. Take the respirator off, press the Power button to activate Standby Mode and don it again as described above.

Join the AirClips on the right hand side

**NOTE!** If you cannot connect the air clips on both sides because the respirator pulls up tight against the back of your neck,

swap to a neck support suited to a larger head size – see above. Then start the donning process again.

Finally, reach behind your head and find the harness rear strap. Locate the hook on the bottom end of the strap. Snap the hook onto the hoop on the filter cover of CleanSpace Respirator so that the rear strap supports the weight of the respirator.

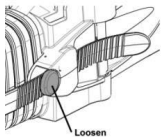
**Adjust the respirator vertical position**  
Adjust the rear strap to set the height of the respirator. CleanSpace Respirator should sit roughly level (as shown below). If your job involves looking up a lot (for instance sanding a ceiling) you may prefer to tighten the strap more to raise the respirator and give yourself more space for head movement.

You can set how close CleanSpace Respirator sits to your neck by adjusting the bellows on either side of the respirator. If your job involves a lot of rapid movement (for instance running up stairs) you may wish the respirator to sit snugly against your neck. If you will be looking up a lot, you may find it more comfortable with the respirator set well back from your neck.

To move the respirator forward: steady the mask with one hand and press on the back of the respirator with the other. You will hear clicking as the adjust mechanism moves to the new position. Be careful to adjust each side by the same amount. To move the respirator backwards, press the adjust buttons on each side and if necessary pull backwards on the respirator.

## 10. Entering and Exiting the Contaminated Area

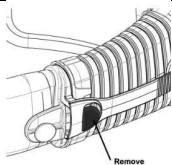
Prior to entering the contaminated area, complete the inspections and checks listed in these user instructions.

1. Ensure Seal Check has been done and motor is responding to your breathing
2. Check that none of the alarms are sounding.
3. Remember that if your workplace is very noisy you may not be able to hear the device alarms. **In this case you must follow the special precautions set out in Section 11.**
4. Enter the work area
5. Do not remove the respirator until you have left the contaminated area, unless you have pressing health reasons to do so (for instance you are experiencing dizziness and believe removing the respirator while you leave the contaminated area may help).

6. Leave the contaminated area immediately if any of the following conditions occur:
  - Any part of the system is damaged
  - Airflow into the mask decreases or stops
  - The battery or filter alarms are triggered (even if only the audible or only the visible alarm triggers)
  - Breathing becomes difficult
  - You feel dizzy or your vision is impaired
  - You smell or taste contaminants
  - Your face, eyes, nose or mouth become irritated
  - You suspect the concentration of contaminants may have reached levels at which this respirator may no longer provide adequate protection.
7. You should have specific exiting and decontamination procedures as part of your workplace respiratory protection program. Follow those procedures, together with the instructions below when removing the respirator.

To remove the respirator, find the Mask Release Button (see Section 5) and press to release the mask from the blower. **The mask release button has a rough, dimpled finish that you can feel with your fingertips. All the other buttons are smooth.**

Note: The respirator motor will automatically switch off and go into Standby Mode in approximately 10 seconds when no breath is detected'

**DO NOT REMOVE THE RESPIRATOR UNTIL YOU HAVE VACATED THE CONTAMINATED AREA unless you have pressing health reasons to do so (for instance you are experiencing dizziness and believe removing the respirator while you leave the contaminated area may help)**

## 11. Working in Noisy Environments

In most noisy environments the CleanSpace Respirator alarms are still clearly audible to the wearer. But if there is a chance that you will not hear the alarms, you must take the following extra precautions

- Never enter the contaminated zone unless all three green battery indicator lights are illuminated
- Never work in the high noise area for more than four (4) hours. At the end of four hours, leave the contaminated zone and check that
  - The filter blocked light is not illuminated; AND
  - All three green battery indicator lights are still illuminated
- If two or fewer green battery indicator lights are illuminated, recharge CleanSpace Respirator until 3 lights are showing again
- If the filter blocked light is illuminated, change the filter
- Be particularly aware of difficulty breathing or of the air flow stopping. If these things happen, exit the contaminated zone immediately.

## 12. Changing the Filter

Before using CleanSpace Respirator you must ensure that you are using the correct filter type for the environment you are working in. The CleanSpace Respirator blower is supplied with a particulate filter.

**IT IS ESSENTIAL THAT THE CORRECT FILTER TYPE IS SELECTED FOR THE CHOSEN APPLICATION.**

Before changing the filter on the respirator, move out of the contaminated area and remove the respirator. Used filters should be disposed of responsibly and treated as non-recyclable hazardous\* waste (\* dependent on the contaminant being filtered). Filters need to be regularly changed. The frequency of change depends on use and concentration of contaminants in the atmosphere.

#### BLOCKED FILTER ALARM

All CleanSpace Respirator respirators have a Filter Blocked alarm, which is triggered when the filter requires replacement (2 beeps, repeated every second).

*If the blocked filter alarm is triggered (2 beeps, repeated every second, red LED flashes), leave the contaminated area immediately and replace the filter.*

*Operating the respirator after the blocked filter alarm has sounded can cause the flow to fall below the manufacturer's minimum designed flow, which may result in overexposure to contaminants and lead to sickness or death.*

(Pressing the Power button once will mute the Filter Blocked alarm, allowing you to concentrate while you exit the contaminated zone. After 15 minutes, if the filter has not been changed, the alarm will resume).

#### WHEN TO CHANGE A PARTICULATE FILTER

Change the filter

- When the Blocked Filter Alarm sounds
- If the Flow Test indicates that CleanSpace Respirator is not able to produce the Minimum Design Flow.
- If there is any sign of damage to the filter
- When the filter reaches its expiry date (marked on the filter label)
- When the outside of the filter is heavily soiled
- When the filter is wet
- If there is any sign of dust or contaminants on the inside surface of the filter.

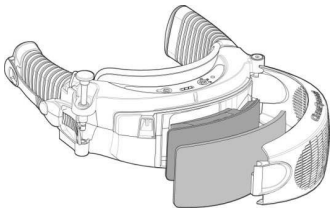
A CleanSpace Respirator with a clean filter will run for much longer than one with a filter that is dirty. To maximise your work time between battery charges, it is best to change the filter more often.

CleanSpace filters are NOT reusable and must NOT be cleaned.

**DO NOT USE COMPRESSED AIR OR ANY OTHER MECHANICAL METHOD TO CLEAN THE FILTER!  
HEPA FILTERS ARE VERY EASILY DAMAGED BY THE USE OF COMPRESSED AIR OR BY BRUSHING.**

*Misuse of CleanSpace filters by improperly cleaning the filter may result in overexposure to contaminants and lead to sickness or death.*

#### HOW TO CHANGE THE PRE-FILTER FOR THE STANDARD P3 FILTER

until clicked back into place.

We recommend changing the pre-filter daily. Prior to opening the filter cover and removing the pre-filter, clean the outside of the respirator with CleanSpace Cleaning Wipes or a damp cloth to prevent potential contamination of the inside of the Power Unit. To unlock the filter cover lift the pin located on the end of the filter cover.

The filter cover should easily swing open exposing the pre-filter. Remove the used pre-filter and replace with a new pre-filter, taking care not to disturb the particulate filter. Swing the filter cover back into its closed position and press the pin down

Figure: Changing the pre-filter (picture shows CleanSpace2 respirator but procedure is the same on all CleanSpace Respirators)

**HOW TO CHANGE THE STANDARD FILTER (PAF-0035)**

To unlock the filter cover lift the pin located on the end of the filter cover. The filter cover will swing open exposing the used filter. Remove the filter and replace with a new particulate filter. Swing the filter cover back into closed position and press the pin down until clicked back into place.

**WHEN TO CHANGE A COMBINATION (LARGE CASE) FILTER**

Change the filter

- Gas filters must be replaced at the first sign of odour, taste or irritation in the mask.
- When the Blocked Filter Alarm sounds
- If the Flow Test indicates that CleanSpace Respirator is not able to produce the Minimum Design Flow.
- If there is any sign of damage to the filter
- When the filter reaches its expiry date (marked on the filter label)
- When the outside of the filter is heavily soiled
- When the filter is wet
- If there is any sign of dust or contaminants on the inside surface of the filter.

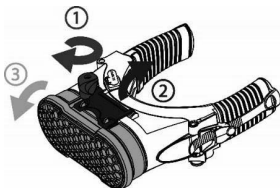
Note that the pleated particulate filter elements in PAF-0050 and PAF-0051 cannot be replaced – instead the whole filter must be replaced.

CleanSpace filters are NOT reusable and must NOT be cleaned.


There is no way to restore the function of the activated carbon in a gas filter. Dispose of the filter and fit a new one.

**HOW TO CHANGE THE LARGE FORMAT FILTERS****STEP 1. Removing the filter from the filter adaptor**

- Turn the respirator upside down. Using your thumb, move the Latch Lock out of the way and then pull the Latch firmly away from the filter. The filter can now be removed.


**STEP 2. Fitting a new filter to the filter adaptor**

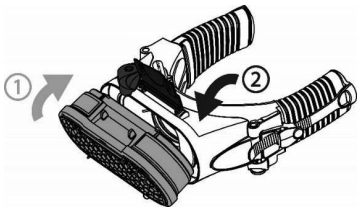- Hold the new filter so that the blue seal is facing the adaptor. Locate 2 ribs on the filter body into the slots in the top of the filter adaptor and push into place. Turn the respirator upside down and push the filter into the adaptor.
- Snap the latch firmly over the 2 ribs on the bottom of the filter body. You will hear 2 clicks when the filter is positioned correctly. **Note: The latch lock will move to one side automatically.**
- The filter is now sealed to the respirator and ready to use

**FITTING A REPLACEMENT FILTER PAD PAF-0043 (AUSTRALIA/NZ ONLY)**

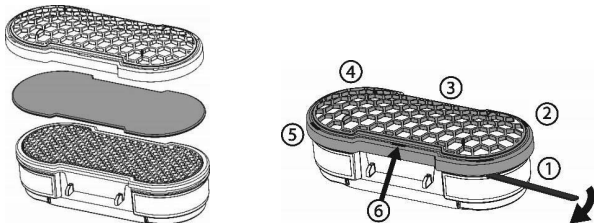For Combination Filters (AUSTRALIA) the particulate filter pad should be changed when the filter alarm alerts the wearer that the particulate filter is blocked.


- Pry the Snap Lid away from the filter using any flat lever in the six places shown in the diagram above.
- Replace with a new particulate filter pad. Ensure that the particulate filter pad is correctly positioned with the text "THIS SIDE TO RESPIRATOR" facing the respirator.
- Replace the Snap Lid and press down firmly at the 6 locations shown until you hear a "click".
- The pad is now sealed to the gas filter and ready to use.

**13. Battery Information**

*Always correctly use and maintain the internal lithium ion battery packs. Failure to do so may result in fire or explosion or could adversely affect respirator performance and result in injury, sickness or death.*

- Do not charge the on-board battery with unapproved chargers, in enclosed cabinets without ventilation, near flammable liquids or gasses, or near sources of high heat
- Do not immerse the device in water
- Do not use, charge or store the device outside the recommended temperature limits

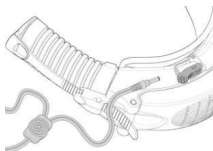Figure 2 - Charging Port - Illustration shows charger with two leads. NOTE: CleanSpace EX charger has only one

**CHARGING THE INTERNAL BATTERY**

*The battery shall only be charged in non-Hazardous areas.*

*Move out of the hazardous area before charging CleanSpace Respirator*

*Charging voltage  $U_m = 14.7V$  from an / via an ATEX certified intrinsically safe Category I (Ex ia) device*

**CleanSpace EX uses a different charger from all other CleanSpace devices. Do not attempt to charge your CleanSpace EX with a charger from a CleanSpace2 or CleanSpace ULTRA. The EX Charger is also not suitable for charging those devices.**

CleanSpace EX requires an AC adaptor with 14.7V nominal output to charge the internal lithium ion polymer battery, which is higher voltage than used in other CleanSpace respirators. **The CleanSpace EX internal battery must only be charged with a CleanSpace Ex charger.**

- **Make sure you are not in an explosive atmosphere.**
- To charge the internal battery, turn the respirator upside down, and locate the charging port. See Figure 2.
- Lift off the flexible polymer charging port cover. Slide the charger cable connector into the charging port. Ensure the charger is plugged into a power socket.
- Charging is complete when the blower shows three (3) green solid lights. If the third green LED is flashing rapidly (80% on, 20% off), charging is 95% complete.
- When the respirator is fully charged, disconnect the charger cable from the blower charging port. **IMPORTANT** Ensure you replace the charging port cover to prevent dirt and contaminants getting into the port.
- **Note:** The CleanSpace Respirator battery can only be charged at temperatures between 0°C and 35°C. Outside this temperature range no charging will occur.

Do not charge the battery in electro-magnetically noisy environments, such as near welding machines. Doing so may damage the CleanSpace Respirator electronics.

Do not leave CleanSpace Respirator plugged into its charger for more than one week at a time. Doing so may shorten the battery life.

#### LOW BATTERY ALARM

CleanSpace Respirator has a low battery alarm, which is triggered when the battery has approximately 5 minutes life remaining (3 beeps, repeated every second). You cannot mute the low battery alarm except by connecting CleanSpace Respirator to its charger or by stopping the motor (doff the respirator and either press the Power Button once or allow CleanSpace Respirator to detect that there is no breathing and stop) When the battery voltage becomes extremely low, the motor will stop.

***If the battery alarm sounds (3 beeps, repeated every second), leave the contaminated area immediately and re-charge the battery. Operating the respirator after the low battery alarm has sounded can cause the flow to fall below the manufacturer's minimum designed flow, which may result in overexposure to contaminants and lead to sickness or death.***

#### GENERAL BATTERY INFORMATION

CleanSpace Respirator has an internal lithium ion (Li-ion) polymer battery. Lithium ion (Li-ion) batteries have the highest energy density of all battery types, and are widely used today in portable electronic devices across many different industries. The commonly available Li-ion polymer batteries are similar to Li-ion batteries, except they are packaged in a soft polymer film as opposed to the metal case commonly used for Li-ion cells. The soft packaging allows the cells to expand slightly under extreme heat, making them safer in fault conditions. CleanSpace Respirators use quality Li-ion polymer batteries and are designed to be viable over a minimum of 500 complete charge and discharge cycles while still maintaining at least 70% of their specified capacity.

#### RESPIRATOR STORAGE

To maintain the integrity of the internal battery, the respirator must be stored under the following conditions when not being used:

Long term storage (> 30 days)	30% - 50% RH, 18°C to 28°C
Short term storage (< 30 days)	30% - 50% RH, -10°C to 35°C

#### The following precautions must be followed during use:

- Avoid mechanical shocks or impacts from any sharp or hard objects
- Do not use or place CleanSpace Respirator in extreme heat, such as in direct sunlight, near heat sources, etc. The battery will be damaged if its temperature rises above 100°C. **Note:** CleanSpace Respirator will stop functioning if its internal temperature rises above +60°C or falls below -10°C
- Do not dispose of CleanSpace Respirator in a fire
- Do not disassemble the blower case, there are no user serviceable parts inside
- Do not use CleanSpace Respirator if there are any signs of severe mechanical damage

***IN THE EXTREMELY RARE CIRCUMSTANCE THAT THE BATTERY IS DAMAGED AND ELECTROLYTE COMES IN CONTACT WITH EYES, FLUSH WITH WATER IMMEDIATELY AND SEEK URGENT MEDICAL ATTENTION***

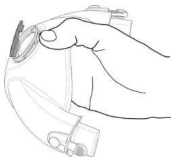
## 14. Cleaning

We recommended that you clean your respirator after every use. The mask, blower, neck support and harness need to be cleaned separately. Disassemble the mask, neck support and harness from the blower before cleaning. CleanSpace cleaning wipes are recommended as a simple and effective method of cleaning the CleanSpace Respirator blower and masks.

### CLEANING THE MASK

There are two methods for cleaning the masks. Before washing the mask, remove the valve cover on the Half Mask or flip up the rain cover on the Full Face Mask to reveal the exhalation valve leaf. To remove the valve cover (Half Mask), find the small inlet located under the lower, inside edge of the valve. Use your finger nail to unclip and push up the valve cover.

The valve cover is easily fitted back by placing it with the clip aligned to the inlet on the valve seating and pushing down. The valve cover will gently snap back into place. See Figure.

#### 1. CleanSpace Respirator Non-Alcohol Wipes for face piece

- CleanSpace Cleaning Wipes are impregnated with benzalkonium chloride, a bactericidal solution ideally suited for silicone face pieces.
- To clean the exhalation valve, remove the valve cover (using the instructions above), gently wipe the leaf and valve seat. Replace the valve cover before use.

#### 2. Hand wash in warm soapy water

- Wash the mask thoroughly with a mild detergent in warm water (less than 50°C).
- A soft brush or sponge can be used to remove any stubborn dirt or grit
- To clean the exhalation valve, gently wash the warm water solution through the valve and use a sponge to gently clean the valve seat
- Rinse the mask and valve well in warm running water. IMPORTANT: If the mask is not rinsed thoroughly, residue from cleaning solution may irritate the wearer's skin or cause the valve to stick. Replace the valve cover before use.

#### Drying the mask

After cleaning, allow the mask to air dry in a clean environment, valve up to prevent water pooling. Note: Do not dry the mask by exposing directly to heat i.e. hair dryers or heaters. The mask can also be hand-dried with a clean, lint-free cloth.

IMPORTANT: Cleaning can cause the exhalation valve leaf to stick. Before use, check that the exhalation valve leaf operates freely by gently lifting the valve. Replace the valve cover before use.

**WARNING: NEVER DRY THE MASK OR EXHALATION VALVE WITH A CLOTH THAT MAY LEAVE BEHIND LINT.**

*Lint contamination of the exhalation valve may cause it to leak, resulting in overexposure to contaminants and lead to sickness or death.*

### CLEANING THE BLOWER

**WARNING: The blower contains a battery, sensitive electronics and a motor. NEVER immerse it in water or use anything wetter than a damp cloth to clean it.**

- After removing the mask, remove the neck support. Leave the filter in place to prevent any dust or liquids from getting into the blower.

**WARNING: NEVER CLEAN THE FILTER WITH COMPRESSED AIR OR A BRUSH. HEPA FILTERS ARE EASILY DAMAGED BY THE USE OF COMPRESSED AIR OR BY BRUSHING.**

*Misuse of CleanSpace Respirator by cleaning the filter may result in overexposure to contaminants and lead to sickness or death.*

- Using CleanSpace Cleaning Wipes, wipe down the outside of the blower. Blowers can be cleaned with a damp cloth.
- Allow the blower to air dry in a clean environment. The blower can also be hand-dried with a clean lint-free cloth. Note: **Do not dry the blower by exposing to heat (i.e. hair dryers or heaters) or compressed air.**

**CLEANING THE HARNESS AND NECK SUPPORTS**

The harness and neck supports can be washed in warm soapy water or machine washed. After cleaning, allow the harness and neck supports to air dry in a clean environment. Do not dry by exposing directly to heat i.e. hair dryers or heaters

**15. Recalibration**

*You must recalibrate the internal pressure sensor any time that your CleanSpace Respirator is exposed to changes in temperature of more than 20°C.*

1. With CleanSpace Respirator in Standby mode (one or more green LEDs lit), press and hold both the Power button and the Filter Test button. *If no LEDs are lit, press the Power button once to enter Standby mode.*
2. When both the blue and red LEDs light, release both buttons.
3. Hold the CleanSpace Respirator still until the red and blue LEDs turn off (10 – 15 seconds)
4. The calibration is complete

**16. Approvals**

*CleanSpace Respirator is a complete respiratory protection system. In addition some combinations of components have been approved for use in some explosive atmospheres. Not all of the components shown in this manual have been approved for use in explosive atmospheres.*

**SEE NEXT PAGE FOR A LIST OF THOSE COMPONENTS THAT HAVE IECEx / ATEX APPROVAL**

**RESPIRATORY PROTECTION APPROVALS (ALL CLEANSPACE RESPIRATORS & ACCESSORIES)**

EN 12942: 1998 + A2:2008

AS/NZS 1716:2012

TM3

PAPR-P2 (with half face masks)

PAPR-P3 (with full face masks)

**WATERPROOF APPROVAL (CLEANSPACE ULTRA AND SEE TABLE BELOW)**

IP RATING 66

**EXPLOSIVE ATMOSPHERES APPROVALS (CLEANSPACE EX AND SEE TABLE BELOW)**

<p><b>IEC60079.0:2012,</b> <b>IEC60079.11:2011</b> Ex ia I Ma, Ex ib IIB T4 Gb</p>	<p><b>Key to IECEx markings:</b> Ex Explosion protected ia Type of ignition protection (intrinsic safety, very high protection) ib Type of ignition protection (intrinsic safety, high protection) I Gas Group (Methane) IIB Gas group Ethylene Ma Equipment protection level for underground mining (very high) T4 Temperature class (Maximum surface temperature 135°C) Gb Equipment protection class (high)</p>
<p><b>EN 60079-0:2012, EN 60079-11:2012</b>  I M1 Ex ia I Ma  II 2 G Ex ib IIB T4 Gb</p>	<p><b>Key to ATEX markings:</b>  Explosive area symbol I Equipment Group (underground mines) II Equipment Group (explosive atmospheres other than underground mines) 2G Equipment category (2 = High level of protection, zone 1. G = gas) Ex Explosion protected ia Type of ignition protection (intrinsic safety, very high protection) ib Type of ignition protection (intrinsic safety, high protection) IIB Explosion group Ethylene T4 Temperature class (Maximum surface temperature 135°C) Gb Equipment protection class (high)</p>

## CleanSpace Respirators

## USER INSTRUCTIONS

### COMPONENTS WITH IECEX AND ATEX APPROVAL

Any combination of these components carries the explosive atmospheres approvals listed above.

#### Power Unit and Filter Cover

Product Code	Description
PAF-0060	CleanSpace EX Power Unit (Full kit, without mask)
PAF-0061	Blower Unit – CleanSpace EX
PAF-0065	Standard Filter Cover – CleanSpace EX

#### Filters – For Particulates

Product Code	Description
PAF-0035	Particulate Filter P3 / TM3 P SL R

#### Masks

Product Code	Description
PAF-0064	Mask: Half Small - EX
PAF-0062	Mask: Half Medium - EX
PAF-0063	Mask: Half Large - EX

#### Accessories & Spares: Harness, Neck Supports & Battery Charger

Product Code	Description
PAF-0073	Harness (for half masks)
PAF-1012	Neck support – Small
PAF-1013	Neck support – Large
PAF-0066	Battery Charger EX

### ACCESSORIES FOR WHICH IECEX / ATEX CERTIFICATION IS NOT REQUIRED

The accessories listed below are used outside the hazardous area and can be used with CleanSpace EX without impacting its explosive atmospheres certifications.

#### Accessories

Product Code	Description
PAF-0032	CleanSpace Cleaning Wipes (100 packs)
PAF-1005	Flow Test Cap (for checking filter condition and battery charge)
PAF-0025	Adaptor for Fit Testing (for half masks)
PAF-1015	Full Face Mask Adaptor for Fit Testing (for Full Face masks)
PAF-0039	Mask Fit Test Kit (Set of half masks with blocked AirClips – for estimating mask sizes)
PAF-1009	Seal Check Cap (for positive pressure seal tests on Half masks)

### COMPONENTS THAT DO NOT HAVE IECEX / ATEX APPROVAL

These components can be added to the system without affecting the respiratory protection approvals listed above. They have not been certified for use in explosive atmospheres.

#### Power Units

Product Code	Description
PAF-0034	CleanSpace2 Power Unit (Full Kit, without mask)
PAF-0070	CleanSpace Ultra Power Unit (Full Kit, without mask)

#### Masks

Product Code	Description
PAF-0033	Mask: Half Small
PAF-1010	Mask: Half Medium
PAF-0027	Mask: Half Large
PAF-1014	Mask: Full Face Mask Medium / Large

## Filters – For Particulates

Product Code	Description
PAF-0037	Particulate Filter P3 / TM3 P SL R High Capacity
PAF-0043	Particulate Filter P3 – Replacement Pads for Combined Filters PAF-0046 and PAF-0047 – 3 Pk (Australia only)
PAF-0053	Particulate Filter P3 – Replacement Pads for Combined Filters PAF-0046 and PAF-0047 – 8 Pk with Honeycomb Lid (Australia only)

## Combined Filters – For Gases &amp; Particulates

Product Code	Description
PAF-0038	Filter Adaptor - all large-case filters (PAF-0037, PAF-0046, PAF-0047, PAF-0050, PAF-0051)
PAF-0046	Combined Filter A1P3 (Australia)
PAF-0047	Combined Filter ABE1P3 (Australia)
PAF-0050	Combined Filter TM3 A1 P SL R (Europe)
PAF-0051	Combined Filter TM3 ABE1 P SL R (Europe)

## Pre-Filters and Coveralls

Product Code	Description
PAF-0036	Particulate Pre-Filter (Standard)
PAF-0057	Particulate Pre-Filter (Large Case Filters)
PAF-0058	Particulate Pre-Filter and Coverall (Standard)

## NOTIFIED BODIES

## Respiratory Protection:

SAI Global Assurance Services Limited (2056)  
PO Box 6236 Milton Keynes MK1 9E

## IECEX

TestSafe Australia (IECEX TSA 13.0024X)  
PO Box 592  
Richmond NSW 2753 Australia

## ATEX

Intertek Testing & Certification Limited (ITS14ATEX27939X)  
Cleeve Road, Leatherhead, Surrey, KT22 7SB  
United Kingdom

## 17. Specifications

<b>Air Flow</b>	Manufacturers minimum design flow: 120 litre/minute <i>Breath-responsive, with max inflow to mask 120 – 200 litre per minute, dependant on altitude, filter and battery condition.</i>
<b>Operating Temperature Range</b>	-10°C to 45°C. <i>The motor will shut down if the temperature of the battery pack rises above 60°C or falls below -10°C</i>
<b>Operating Humidity Range</b>	Zero to 90%, non-condensing
<b>Operating Altitude Range</b>	Approximately sea level to 1200m
<b>Charging Temperature Range</b>	0°C to 35°C <i>Below 0°C and above 35°C the battery may not accept charge</i>
<b>Storage Conditions</b>	<b>Short term storage under 30 days:</b> Out of direct sunlight, in a clean, dry environment (30% - 50% RH), -10°C to 35°C. <i>To maintain the integrity of the internal battery, the respirator must be stored under the following conditions when it will not be used for longer than 1 month</i> <b>Long term storage (&gt; 30 days):</b> Out of direct sunlight, in a clean, dry environment (30% - 50% RH), 18°C to 28°C.
<b>Shelf Life (when stored as specified)</b>	<i>We recommend to charge the battery every 6 months if the device is not in use.</i> Particulate Filters: 3 years

	Gas and Combined Filters: 3 years
<b>Battery Type</b>	Lithium Ion Polymer
<b>Run Time</b>	Up to 8 hours. Certified to a minimum 4 hours. <i>Run time is strongly influenced by work rate, filter condition, altitude and other factors. The run time specified above assumes moderate work rates at sea level with a clean filter. Different conditions will produce widely varying run times.</i>
<b>Recharge Time</b>	2 hours (to 95%)
<b>Low Battery Voltage Alarm</b>	Triggers when the remaining run time is around 5 to 10 minutes. <i>Audible alarm, 3 beeps, repeated every second, 75dB(A) at ear.</i>
<b>Filter Blocked Alarm</b>	Triggers when the filter requires replacement. <i>Audible alarm sounds until a key is pressed, 2 beeps repeated every second, 75dB(A) at ear plus red filter LED flashes. When the alarm first sounds, and if the battery is almost completely run flat, the maximum flow that can be delivered has been reduced to about 140 litre/minute.</i>
<b>Charger</b>	Input: 100 – 240VAC, 50 – 60 Hz. Output: 14.7 VDC, 24 Watts
<b>Intrinsic Safety</b>	CleanSpace EX is certified to IECEx and ATEX standards. See Approvals above.

## 18. Product Warranty

This product has been manufactured using quality parts and processes.

PAFtec Australia Pty Ltd warrants that the product is free from defective workmanship and parts for a period of one (1) year from the date of original purchase provided the product has been used, cleaned and maintained in accordance with these instructions and PAFtec's recommendations. This warranty does not include consumable parts, such as filters and face masks, which must be replaced regularly by the user. Consumable parts are warranted up to the point of use, provided they have been stored correctly and are within their expiry date. This warranty does not cover:

- Where the product has been used for industrial purposes outside the recommendations of PAFtec Australia Pty Ltd;
- Where damage has been caused by misuse, neglect, accident, or excessive wear and tear.

Any claim under this warranty must be made within one (1) year of the date of purchase of the product. All warranty claims must be made by returning the defective product to your supplier together with the proof of purchase. The purchaser is responsible for all freight. In the event that any part of the product is found by PAFtec to be defective, PAFtec will either repair or at its discretion replace the faulty part.

This warranty is provided in addition to other rights and remedies you have under law. You are entitled to replacement or refund for a major failure. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

This warranty is given by:

PAFtec Australia Pty Ltd ABN 24 146 453 554  
 GF 16-18 Carlotta St Artarmon NSW 2064 Australia;  
 T. +612 8236 4000 E. [customerservice@paftec.com](mailto:customerservice@paftec.com)

### Disclaimer:

Whilst Paftec has made every effort to ensure that the details and information given in both our printed and online publications are accurate at the time of issue, full technical specifications are not necessarily included. Furthermore Paftec has a policy of continuous improvement and the right is reserved to alter details and information as the need arises. Accordingly the Customer should check any details and information they wish to rely on with Paftec at the time of purchase. Paftec cannot accept liability in respect of any errors or omissions herein contained or for any loss or damage malfunction or consequential loss arising from reliance upon our publication. The Customer will be responsible for any risk to health or safety from goods in the Customer's possession and/or control. The Customer's attention is drawn to the fact that statutory regulations and recognised codes of practice exist covering the use and handling of some goods (including safety products). The Customer must ensure that persons who use the goods receive adequate training and safety literature. No page or part of this manual may be reproduced in any form without written permission of the copyright owner shown above. Paftec reserves the right to correct typographical errors. All information is correct at the time of going to print.

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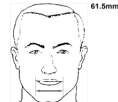
FACE LENGTH	34.5mm	43.5mm	52.5mm	61.5mm
133.5mm	First Try: <b>MEDIUM*</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM*</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>	
123.5mm	First Try: <b>MEDIUM</b> Then Try: <b>SMALL*</b>	First Try: <b>MEDIUM</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>	
113.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	
103.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	
93.5mm				
	<b>LIP LENGTH</b>			

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FACE LENGTH	133.5mm	First Try: <b>MEDIUM*</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM*</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>	
	123.5mm	First Try: <b>MEDIUM</b> Then Try: <b>SMALL*</b>	First Try: <b>MEDIUM</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>	
	113.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	
	103.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	
	93.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	
		34.5mm	43.5mm	52.5mm	61.5mm
		LIP LENGTH			

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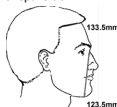
FACE LENGTH	LIP LENGTH		
	34.5mm	43.5mm	52.5mm
133.5mm	First Try: MEDIUM* Then Try: LARGE	First Try: LARGE Then Try: MEDIUM*	First Try: LARGE Then Try: MEDIUM
123.5mm	First Try: MEDIUM Then Try: SMALL*	First Try: MEDIUM Then Try: LARGE	First Try: LARGE Then Try: MEDIUM
113.5mm	First Try: SMALL Then Try: MEDIUM	First Try: MEDIUM Then Try: SMALL	First Try: MEDIUM Then Try: SMALL
103.5mm	First Try: SMALL Then Try: MEDIUM	First Try: SMALL Then Try: MEDIUM	First Try: SMALL Then Try: MEDIUM
93.5mm	First Try: SMALL Then Try: MEDIUM	First Try: SMALL Then Try: MEDIUM	First Try: SMALL Then Try: MEDIUM

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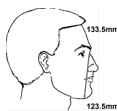
<b>FACE LENGTH</b>	133.5mm	First Try: <b>MEDIUM*</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM*</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>
	123.5mm	First Try: <b>MEDIUM</b> Then Try: <b>SMALL*</b>	First Try: <b>MEDIUM</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>
	113.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>
	103.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>
93.5mm	34.5mm	43.5mm	52.5mm	61.5mm
	<b>LIP LENGTH</b>			

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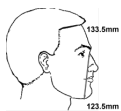
<b>FACE LENGTH</b>	133.5mm	First Try: <b>MEDIUM*</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM*</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>	
	123.5mm	First Try: <b>MEDIUM</b> Then Try: <b>SMALL*</b>	First Try: <b>MEDIUM</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>	
	113.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	
	103.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	
	93.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	
		34.5mm	43.5mm	52.5mm	61.5mm
		<b>LIP LENGTH</b>			

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FACE LENGTH	LIP LENGTH		
	34.5mm	43.5mm	52.5mm
133.5mm	First Try: <b>MEDIUM*</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM*</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>
123.5mm	First Try: <b>MEDIUM</b> Then Try: <b>SMALL*</b>	First Try: <b>MEDIUM</b> Then Try: <b>LARGE</b>	First Try: <b>LARGE</b> Then Try: <b>MEDIUM</b>
113.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>	First Try: <b>MEDIUM</b> Then Try: <b>SMALL</b>
103.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>
93.5mm	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>	First Try: <b>SMALL</b> Then Try: <b>MEDIUM</b>

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- Om produkten har använts för industriella ändamål utanför rekommendationerna från PAFtec Australia Pty Ltd.