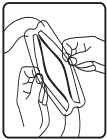
# FFP2 FISH

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**EN-**Cup the mask in your hands with the nose clip facing up.

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**EN-**Position the mask under your chin and pull the straps behind your ears.

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**EN-**Place two fingers on the middle of the nose clip. Mould the clip to the shape of your nose. Moulding with only one finger may lead to a bad seal of the mask.

Before entering a contaminated area, you must test the seal of the mask.

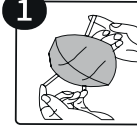
a) Cover the front of the mask with both hands, while avoiding to move the mask.

b) Sharply exhale and inhale. If air leaks around the nose, readjust the nose clip as described in step 3. If air leaks from the mask edges, repeat steps 1 to 3 until the mask fits properly.

c) If no air leaks, the mask is ready to be worn.

# FFP2

①

1. Place the mask over the nose and mouth, under the chin with the nose facing upwards.

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③

3. Using the fingertips, push the nosepiece from the outside to the inside to adhere it to the nasal septum. During this trial phase of the mask the user may be faced with less oxygen flow.

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④

4. Before entering the work area, do several breathing movements to check the mask for leaks or drafts.

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⑤

5. If there are no air leaks or leak inlets, you can continue to enter the contaminated area with the mask.

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