INSTRUCTION MANUAL

NEWSTAR

products

FPMA-HAW200

Medical Wall mount

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Medische steun


Support de médical

Supporto medicali

Soporte médico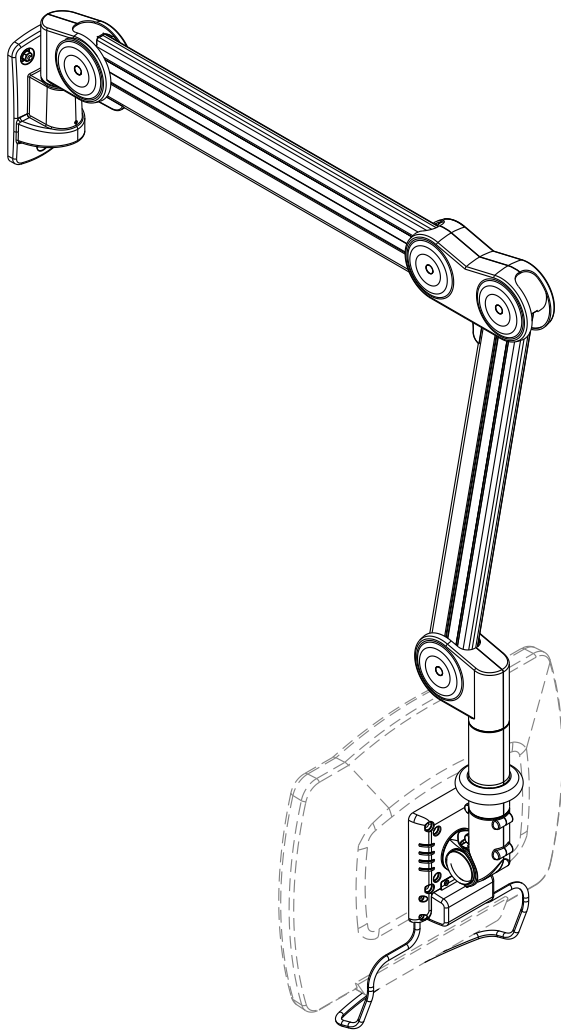

Before starting Assembly

- Please spend some time reading this installation guide then follow the simple step by step guide to assembly.
- Some panels are viewed differently from the exploded drawing below, this is for extra clarity.
- Do not discard any of the packaging until you have checked all the parts and the pack of fittings.
- If you are missing any of the parts, contact supplier where you purchased.

THE INSTALLATION SHOULD BE PERFORMED BY A QUALIFIED TECHNICIAN AND SHOULD MEET THE REQUIREMENTS OF LOCAL REGULATIONS.


Cautions



- Once assembled DO NOT dis-assemble.
- Make sure that mounting screws are suitable for the vertical surface and adequate to hold the weight of the arm and the monitor.
- Support the monitor and arm throughout assembly and mounting.
- Keep small parts out of the reach of children.

- Make sure that the vertical surface is substantial to take the weight of the arm and monitor.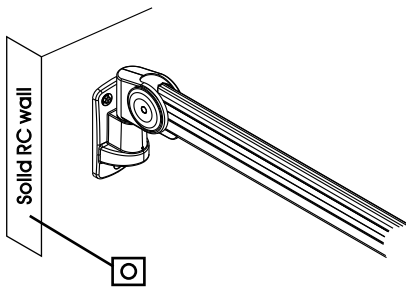

- Only use with LCD/TFT monitors that are VESA compliant- 75/75mm, 100/100mm.

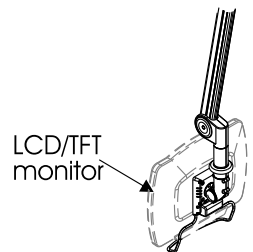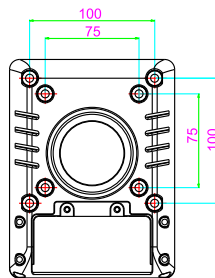
- Monitors to be mounted should not weigh any more than between.


Loading: 1 ~ 7 kg. **(The loading has set up at 7 kg in Factory.)**

- The product's loading capacity is based on monitors' center of gravity less than 3cm from the end of the VESA interface.
- Loading capacity can be reduced if the monitor size is greater than 26 inches or the depth is greater than 5.5cm.
- All products specifications in catalogues are subject to change without notice.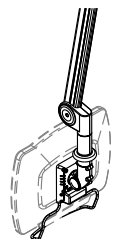


Arm Operation

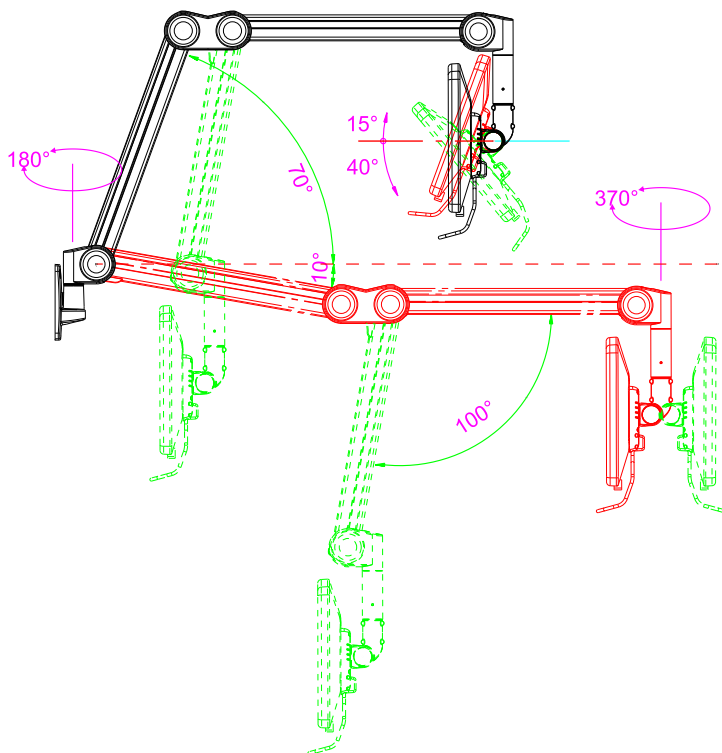
The serviceable range of arm

Top view unit:mm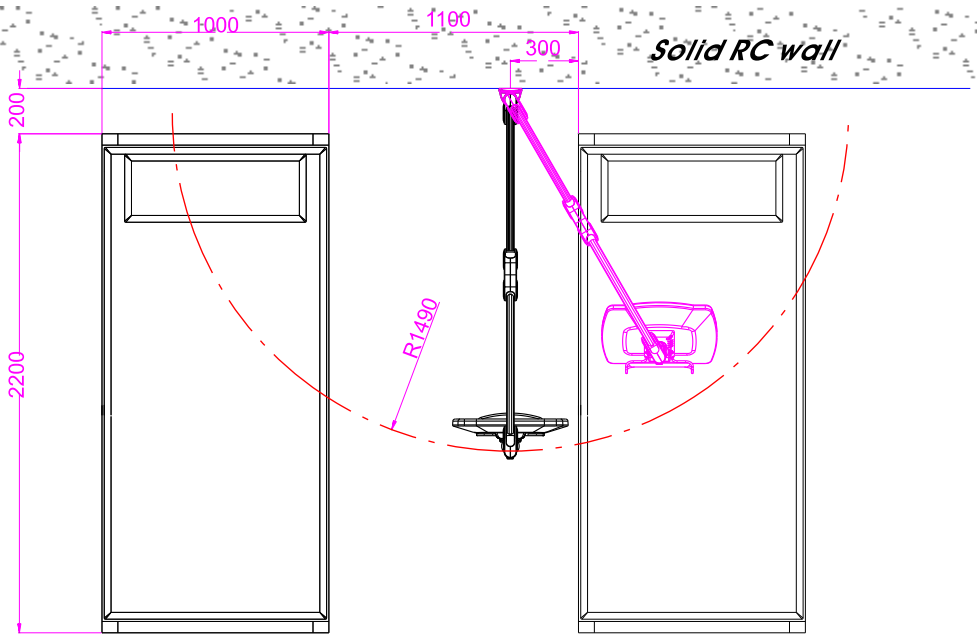


Right view

Unit:mm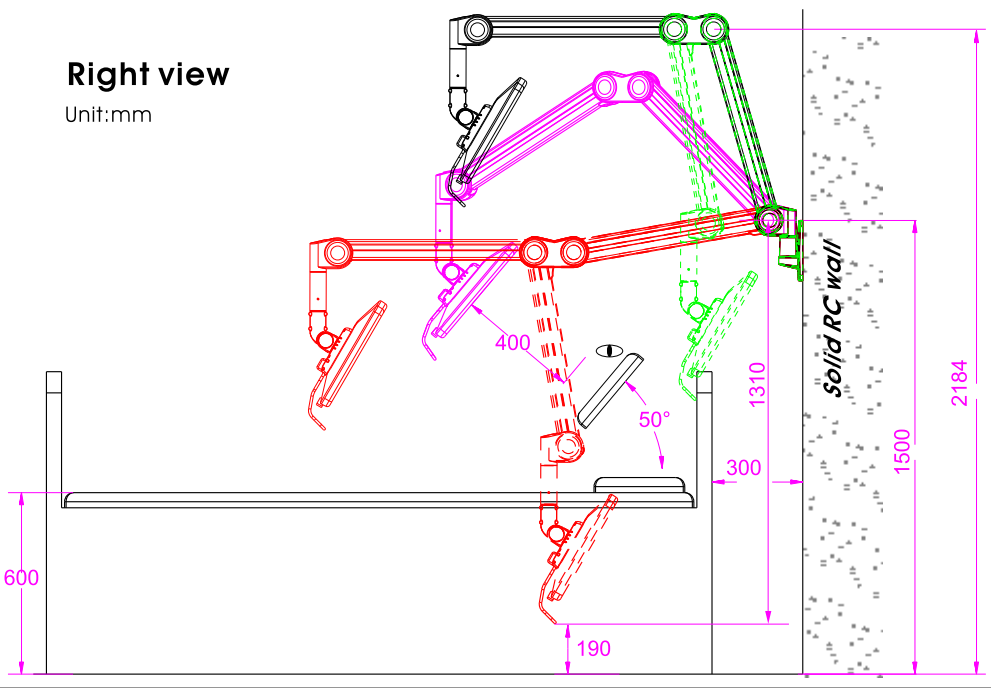


Check list

01 Arm X1

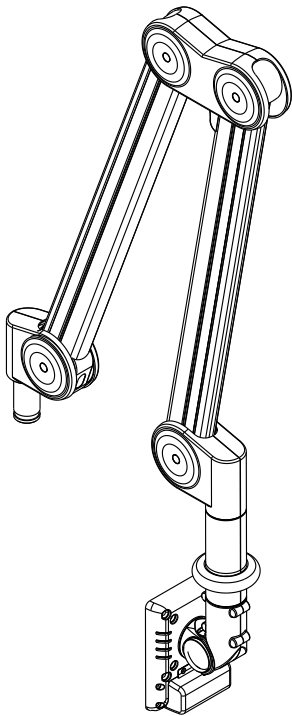
02 Grab Handle X1

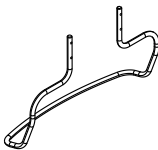03  R-pin X1

04  Cable bolt X1
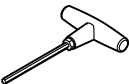
05  "T" Allen key X1

06  (3mm) Allen key X1

07  Screw cover X8


08  (M4X12) Screw X4


09  (M6X30) Screw X4

10  (M4X25) Screw X4

11  spacer sleeve X4


12  (2.5mm) Allen key X1

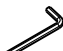13  (4mm) Allen key X1

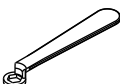14  Wrench X2


15  VESA cable cover-B X1

16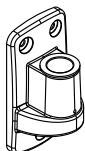

Wall mount X1

Part introduction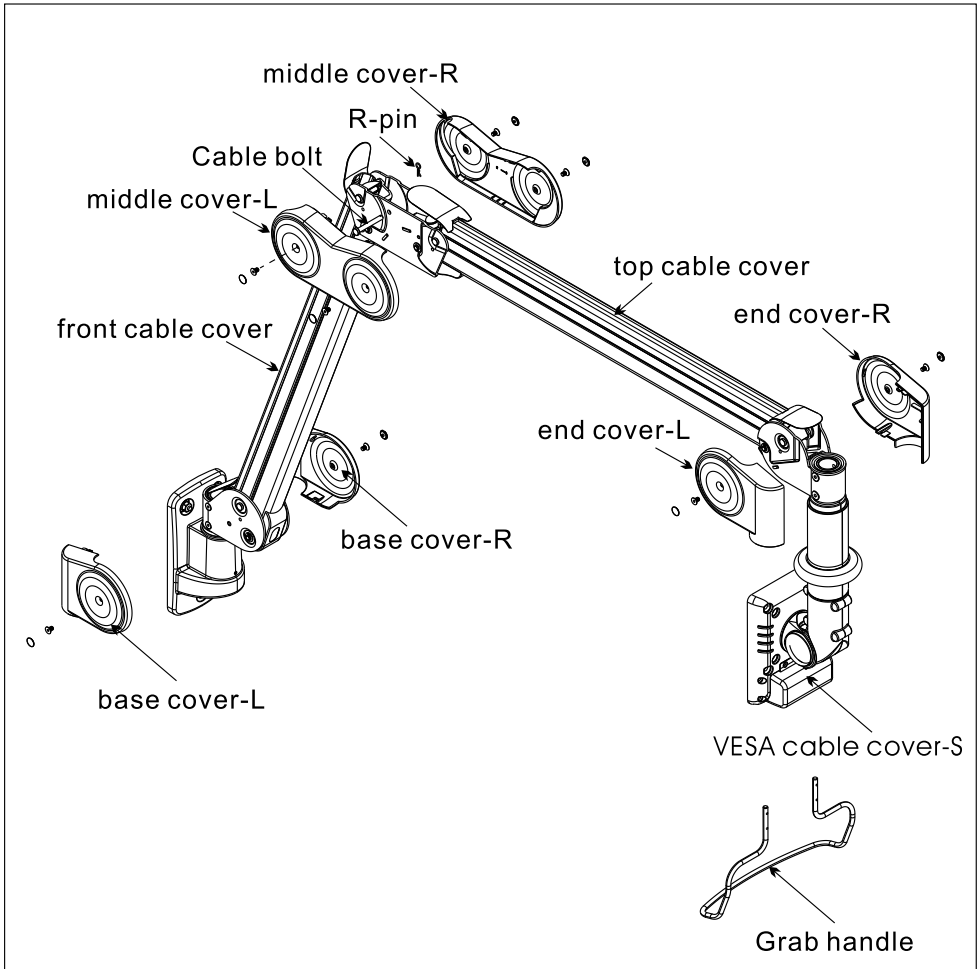

Installation Procedure

Step 1 : Wall mount installation

Step 2: Attach the medical monitor arm to wall mount

Step 3: Cable Assembly

Step 4: Attach Monitor to VESA, Complete the circuit







Step 5: Fixing the Grab Handle

Step 6: Adjustment for Loading

Step 7: Replace the arm covers



Note:

●Use level gauge or protractor to help the wall mount installation.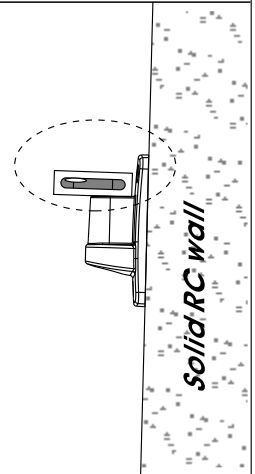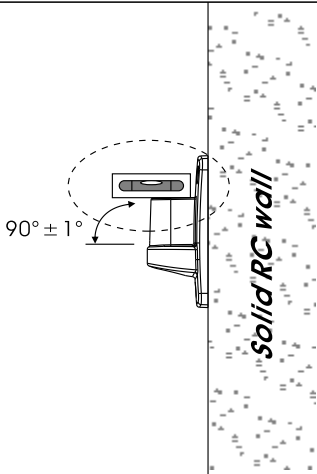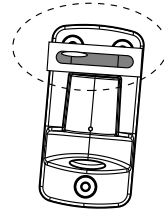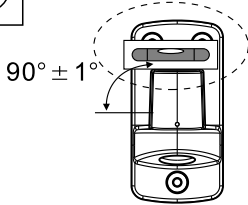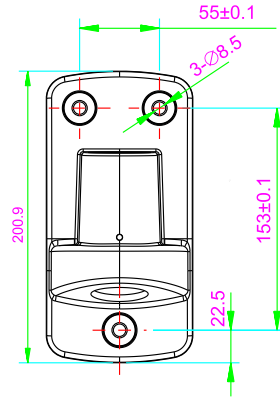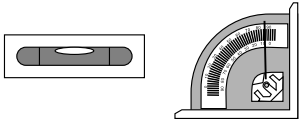

STEP 1. Wall mount assembly



- Make sure the substantial wall before assembling.
- Put the wall mount on the exact place & height that you want and have 3 marks on the wall.
- Install the wall mount on the wall with 3 expansion bolts(not provided).
- Make sure that the wall that the wall mount attach to is vertical.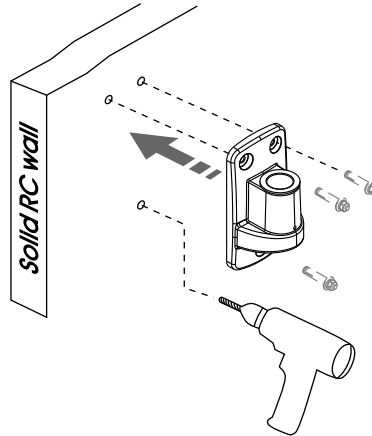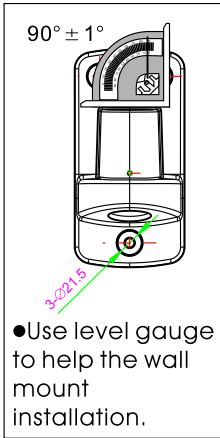

STEP 2.

Make sure the plastic bushing already include in the wall mount before put the arm into the wall mount.

Use 2.5mm allen key to secure the pole and the wall mount with the screw when feel resistenz, loosening the screw half turn.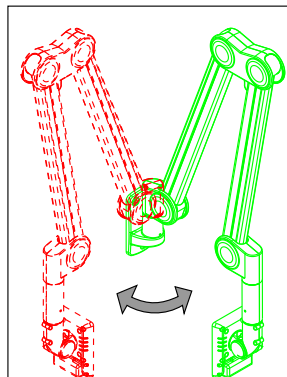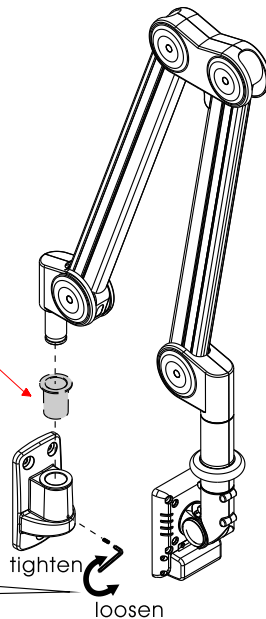


Move the arm from side to side to verify the arm moves correctly.

STEP 3. Cable assembly


3.1 Remove arm covers :

1. Screws on the covers and keep the screws for replacement later.
2. Remove the covers in the way as instructed in the figures.


●Use 3mm allen key to loosen the screw.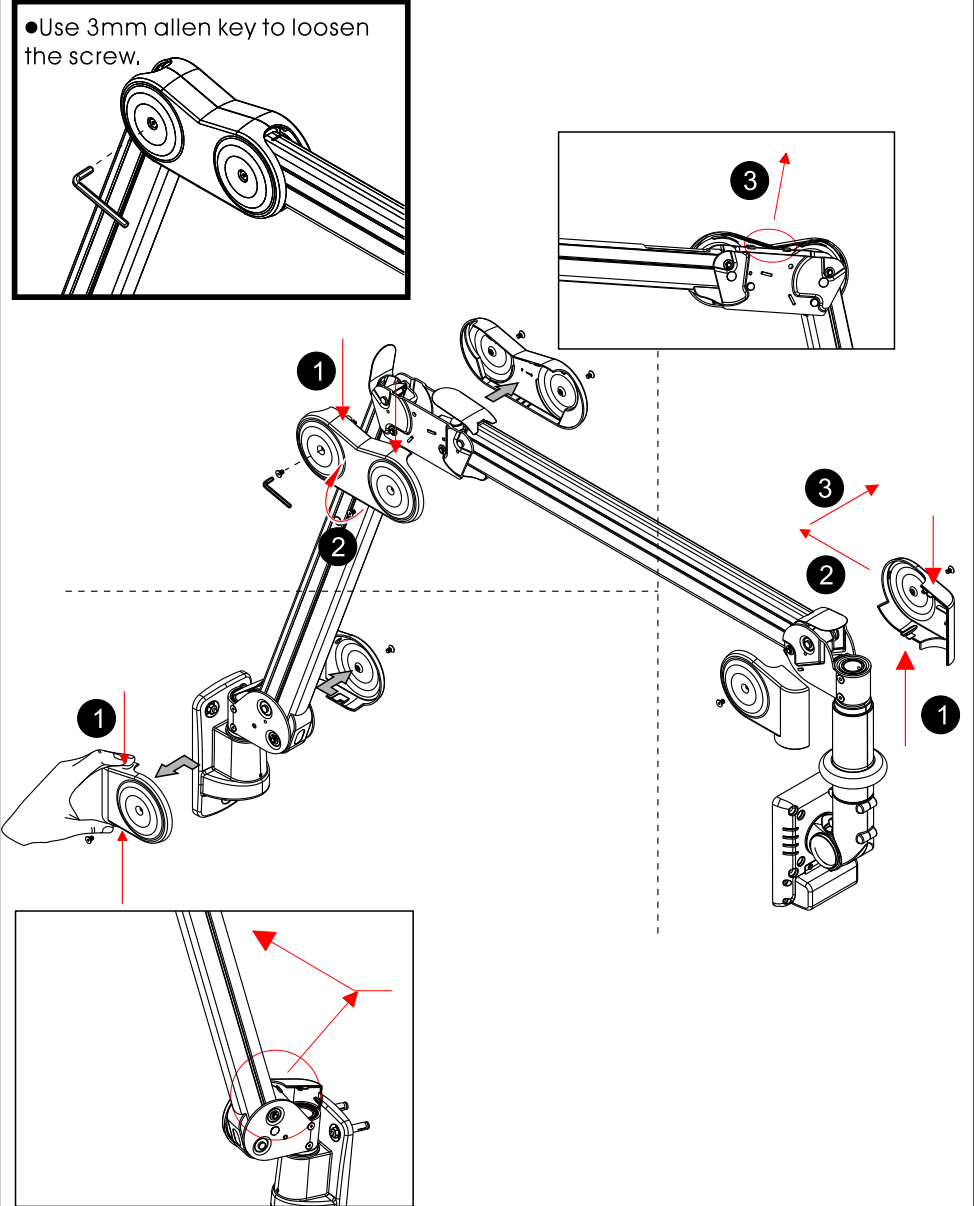


3.2 Remove cable covers :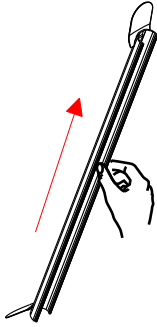

Slid the cable cover out of the arm slot carefully as shown.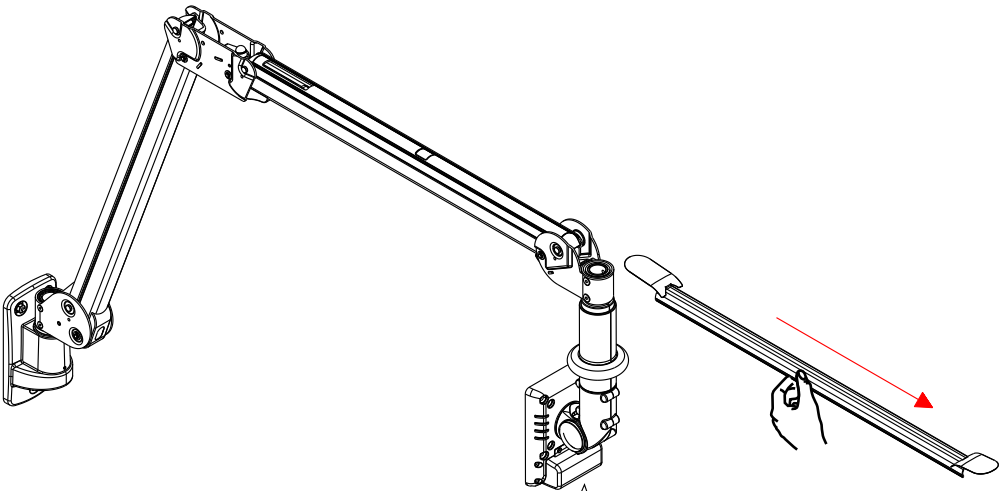

- Remove the VESA cable cover by the screw driver (not provided) as shown in figure. keep the screws for re-screwing after the monitor installation are completed.

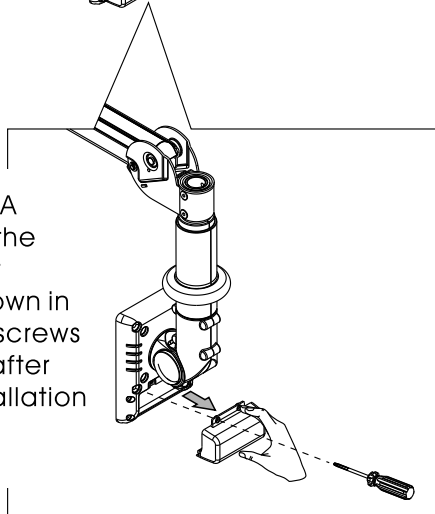


3.3 Insert cables through arm

Use the cable bolt and R-pin to fix the cable with wire loom. (as figure).

The cable with wire loom must lay under the cable bolt!

Some slack must be left in the cables to allow for arm movement.

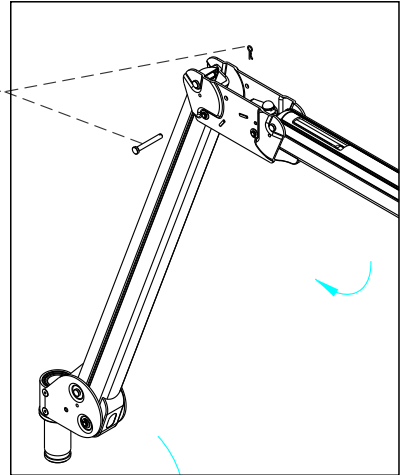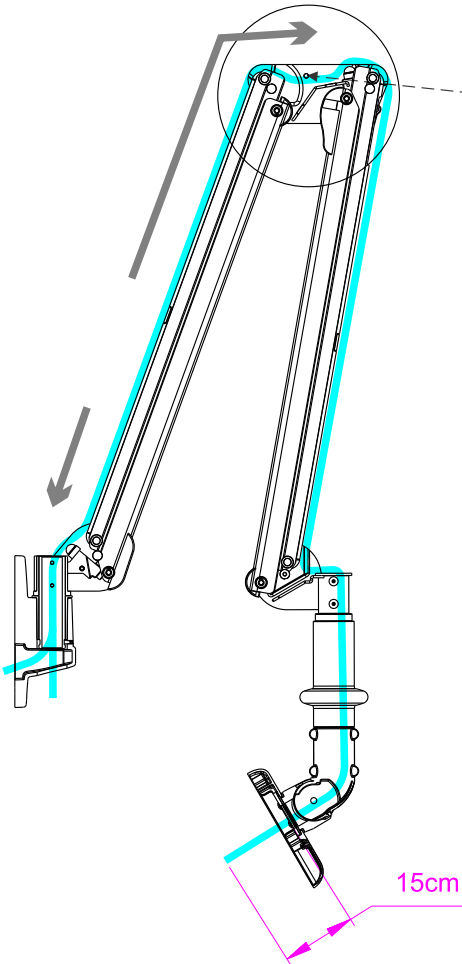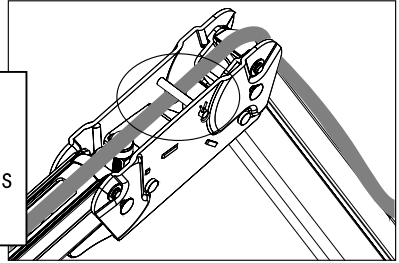
●The VESA adapter must be moved fully down for putting the cable through.

●Allowing for arm movement, we suggest leave at least 15 cm long cable from the VESA adapter as shown.



STEP 4. Attach Monitor to VESA, Complete the circuit

4.1 Fix the Monitor on the VESA mount using Allen Key and screws.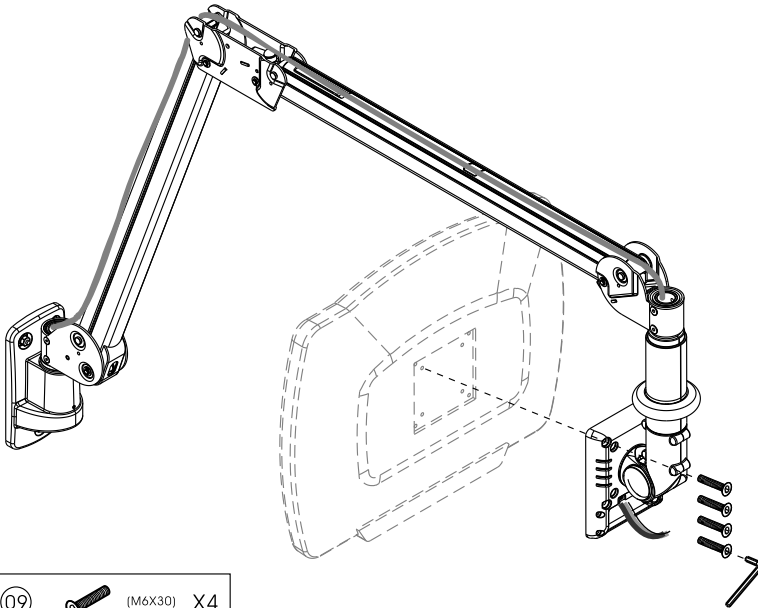

- | | | | |
|----|--|---------|----|
| 09 | | (M6X30) | X4 |
| 13 | | (4mm) | X1 |



- | | | | |
|----|--|---------|----|
| 10 | | (M4X25) | X4 |
| 11 | | | X4 |
| 12 | | (2.5mm) | X1 |

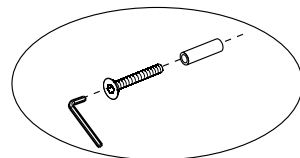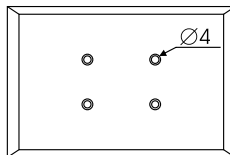4.2



Connect cable to monitor

Connect the cables to the monitor, complete the circuit and then replace the VESA cable cover-B or VESA cable cover-S.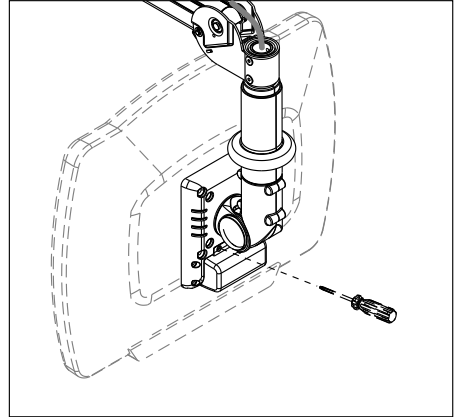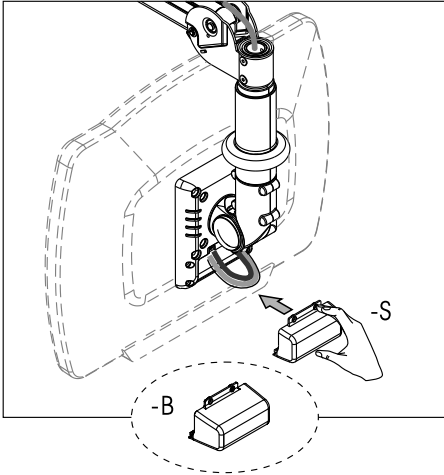

STEP 5. Fixing the Grab Handle

- Insert the grab handle into the 2 holes on the bottom of VESA interface.
- Then fix the grab handle with 4 screws by 2.5 mm Allen Key through the 4 holes on the lower part of VESA interface.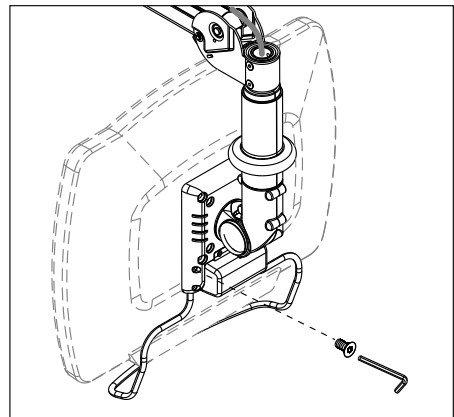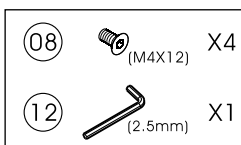

STEP 6. Adjustment for loading

Read the entire steps before starting the adjustment!


Caution for weight capacity adjustment:

Loading : 1~7 kg.(The loading has set up at 7kg in Factory.)


- The product's loading capacity is based on monitors' center of gravity less than 3cm from the end of the VESA interface.
- Loading capacity can be reduced if the monitor size is greater than 26 inches or the depth is greater than 5.5cm.

6.1

- Make sure you have checked out the weight of the monitor you wish to mount .
- Support the arm throughout/during assembly.
- Remove the adjuster covers as shown.

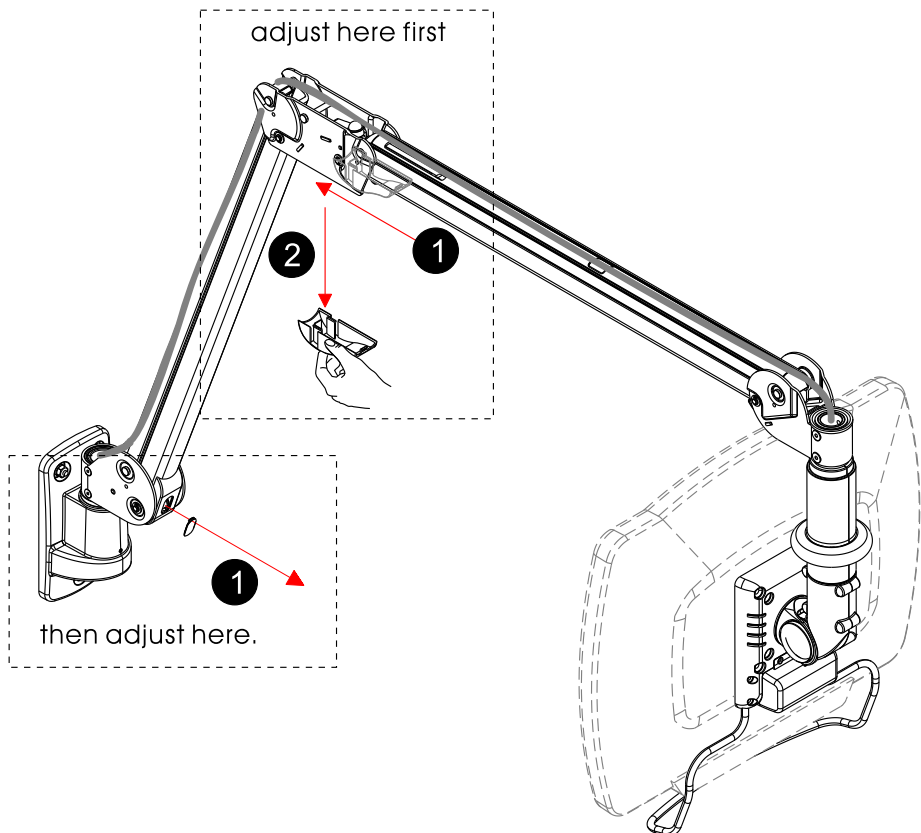
6.2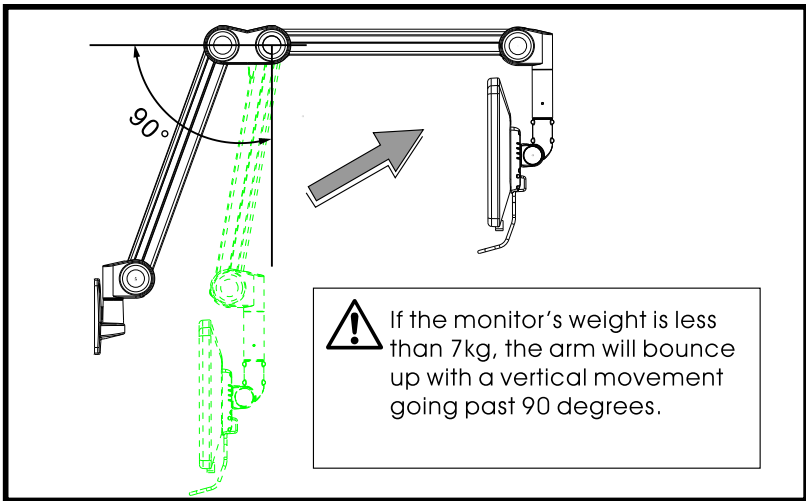


Adjustment:


1. Adjust the arm close to the monitor first.
2. Use the T-Allen key provided to adjust the weight capacity.
3. Turn the T-Allen key clockwise for more weight; counterclockwise for less weight.
4. Continue the process until the desired adjustment is reached.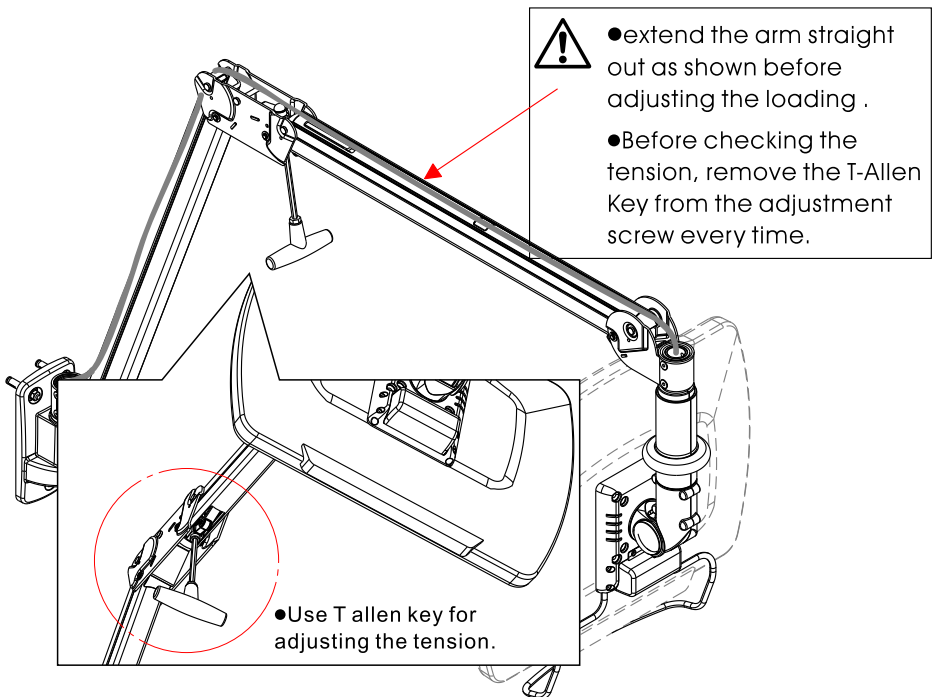

6.3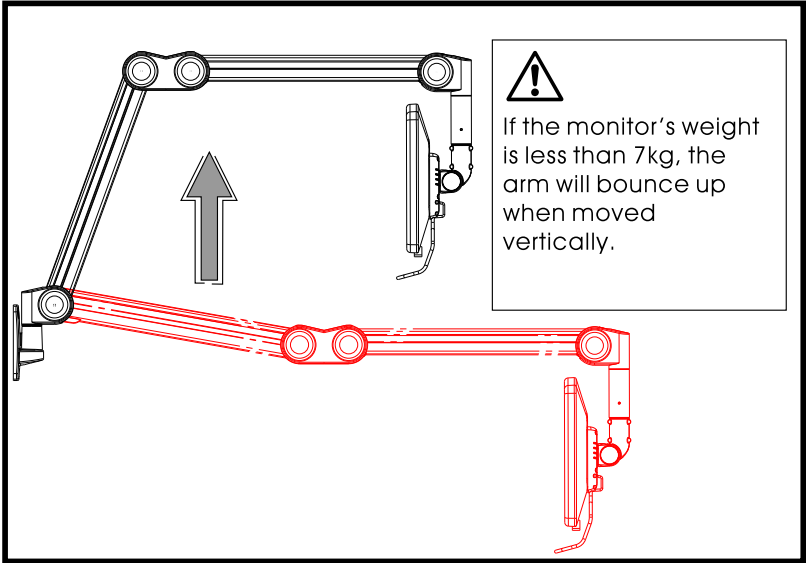

●Extend the arm straight out before adjusting the load.


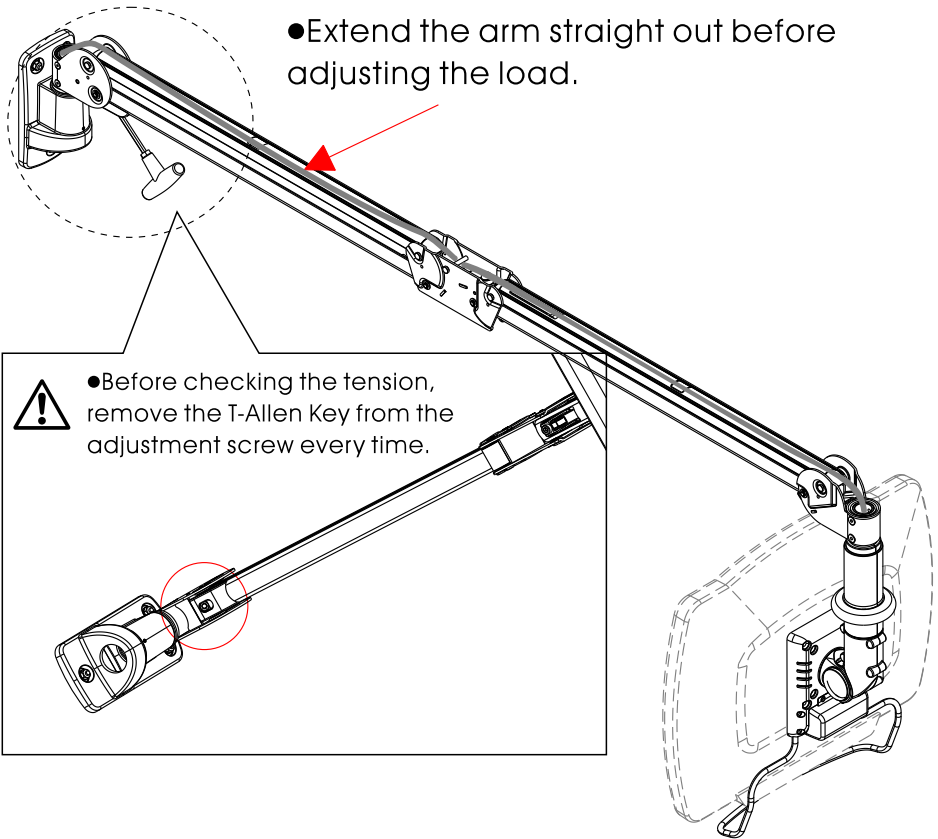


STEP 7. Replace the arm covers

7.1 Follow the order as below to assemble the covers.

1. Replace the adjuster covers as shown.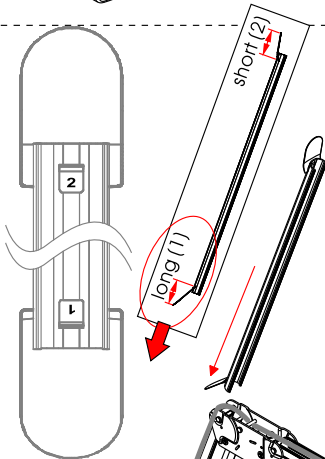
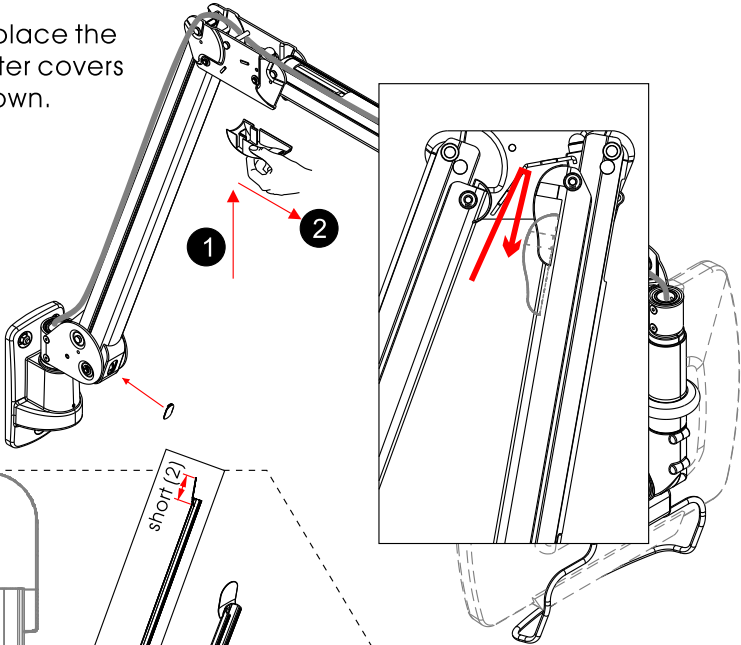




2. Slide the cable covers into the slot as shown carefully. The numbers are indicated on the inner side.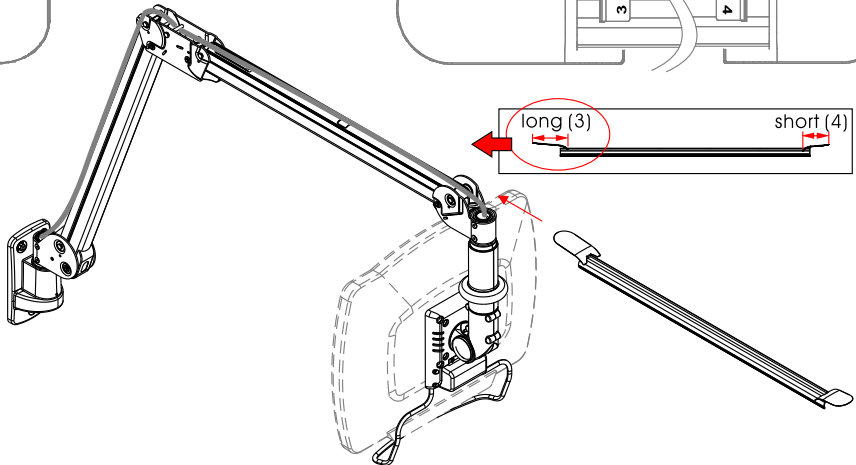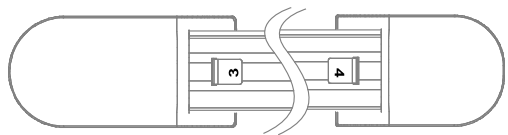

7.2




- 1 Replace the middle covers.
The arrow inside the cover indicates the monitor direction.
- 2 Press down the cover and click it on the metal part!
- 3 Insert the white covers inside the middle cover before attaching the other piece.
- 4 Attaching the other middle cover to and press them together.
Please make sure the cover hooks have got together before pressing.
- 5 Screwing the covers with metal together then take small screw cover to cover the screw hole.

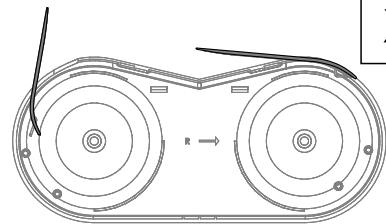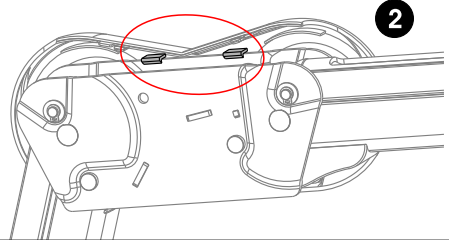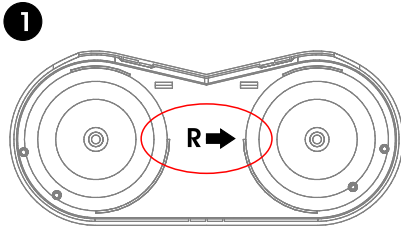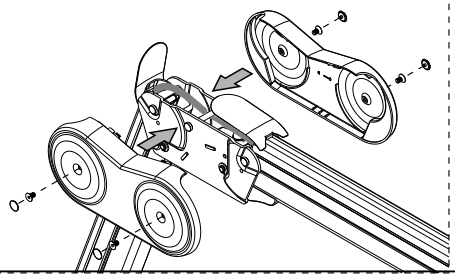X



Please note the middle covers have to hold the white cover.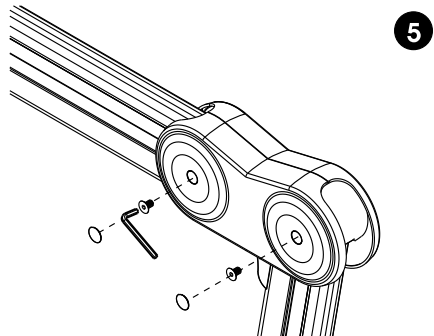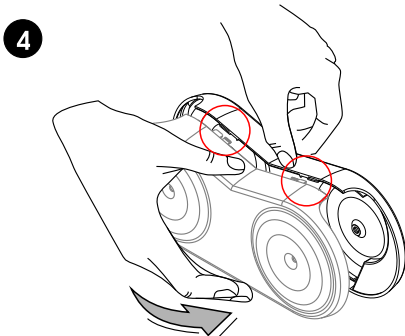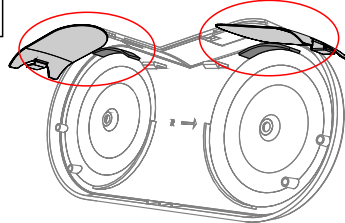


7.3

● Press the arm downward before starting following installation.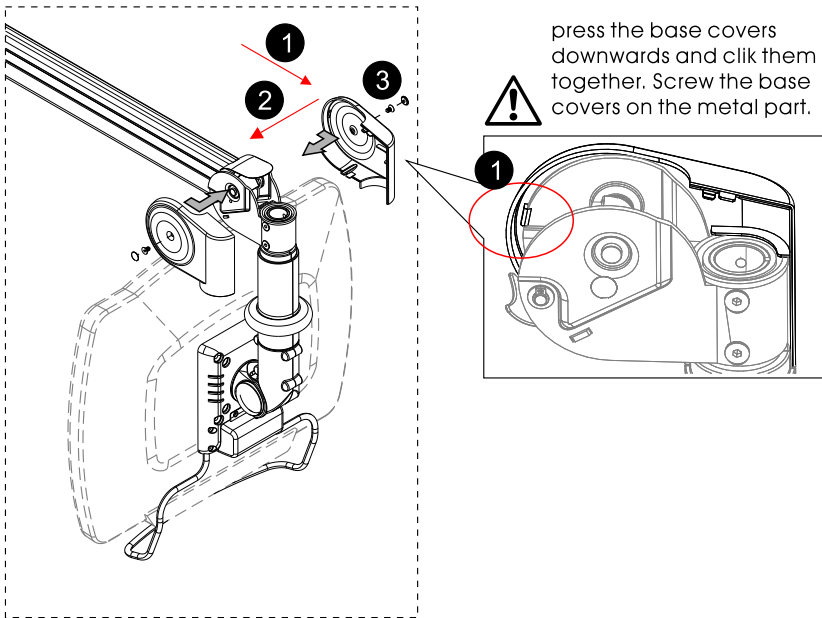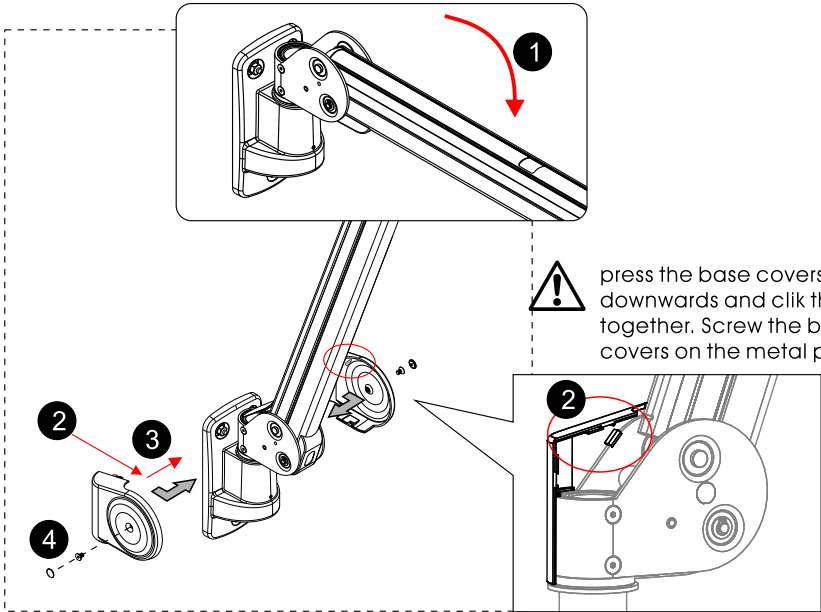

Adjusting Universal Joint Tension

When assembling the arm with monitors of different weights, you may find that the tension of the universal joint needs to be adjusted to ease movement or to maintain the desired monitor position. If so, adjust as follows: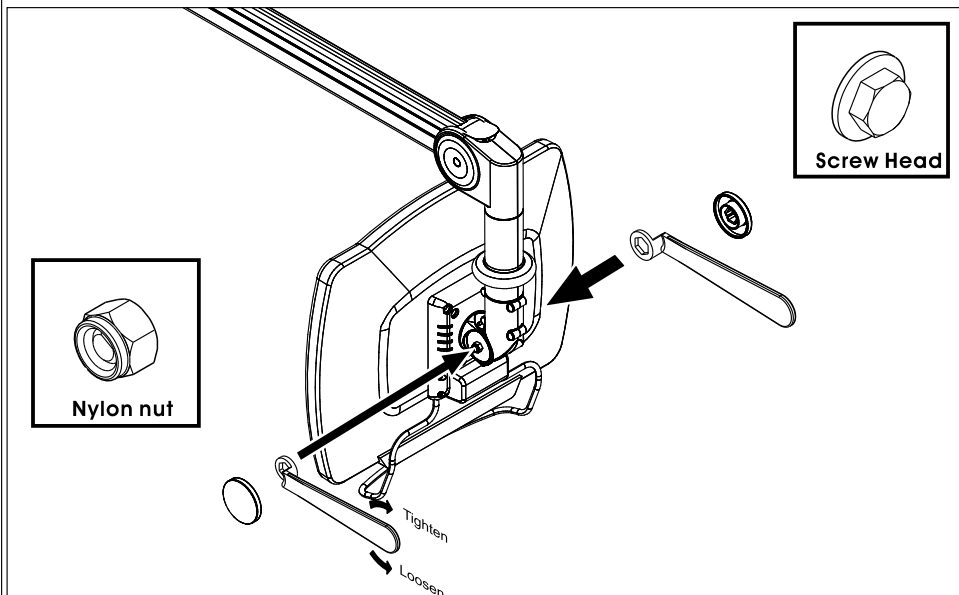

- Remove the plastic universal joint cover.
- Find the nylon nut (not the screw head) and use the provided wrench to adjust the tension. Fix the screw head by wrench at the same time when adjust.
- Use one hand to support the monitor and adjust the tension of the universal joint.
 - Clockwise : more tension
 - Counterclockwise : less tension
- When the desired tension is achieved, replace the plastic cover.


Caution:

- Support the monitor during the adjustment procedure.
- Only adjust the nylon nut (not the screw head).
- Do not disassemble the universal joint.


Safety Notes


- Do not hang on the arm.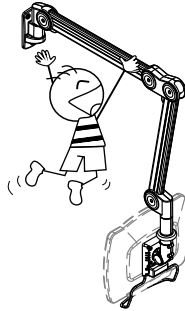

- Do not use the arm as support .

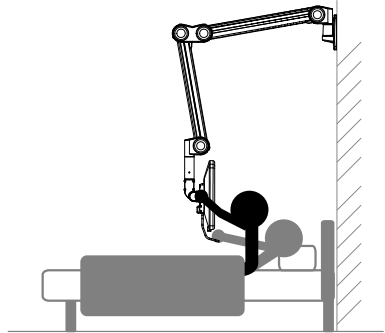- This arm is for supporting Monitors of weights between 1 and 7kg only. Do Not use this arm for holding other products.

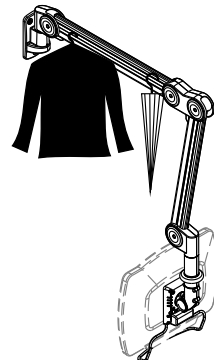EN Caution

1. To ensure safety, please read this manual carefully before installation and follow the instructions. Store this manual in a secure place for future reference.
2. The manufacturer shall not be legally responsible for any equipment damage or personal injury caused by incorrect installation or operation other than that covered in this manual.
3. The mount is designed for easy installation and removal. The manufacturer shall not be liable for damage to equipment or personnel injuries arising out of human factors or acts of God, such as earthquake or typhoon.
4. It is recommended that the Wall mount bracket be installed by qualified personnel only.
5. At least two persons are needed to install or remove the product to avoid hazard of falling objects.
6. Please carefully inspect the area where the wall mount is to be installed:
 - Avoid places that are subject to high temperatures, humidity or contact with water,
 - Do not install the product near air conditioning vents or areas with excess dust and fumes.
 - Only install on vertical walls and avoid slanted surfaces.
 - Do not install in places subject to any shock or vibration.
 - Do not install in places subject to direct exposure to bright light, as it may cause eye fatigue when viewing the display panel.
7. Maintain sufficient spaces around the display to ensure adequate ventilation.
8. To ensure safe installation, first check the structure of the wall and select a secure mounting location.
9. The wall should be strong enough to sustain a weight of at least four times of the display and Wall mount bracket combined. The mounting location must be able to withstand earthquake or other strong shock.
10. Do not modify any accessories or use broken parts. Contact your dealer with any questions.
11. Tighten all screws (do not use excessive force to avoid breaking the screw or damaging its thread).
12. Drill holes and bolts will be left in the wall once the display and Wall mount bracket are removed. Stains may occur after extended use.
13. Since the manufacturer has no way to control the wall type and installation of wall mount, the warranty of the product shall only cover the body of the wall mount. The warranty period of the product is 5 years.
14. Please consult the English language manual for any dispute on conditions.

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