- Ⓓ Montage- und Gebrauchsanleitung
- ⒼⒷ Installation and Operating Instructions
- Ⓕ Notice de montage et mode d'emploi
- ⒼⓁ Installatie- en gebruikshandleiding
- Ⓘ Istruzioni di montaggio e per l'uso
- Ⓔ Instrucciones de instalación y funcionamiento
- Ⓐ Instrukcja montażu i obsługi

(D) HEWI Support	0049 5691 82-300 (D), 00 800 4394 4394 (A, CH Freecall Festnetz)
(GB) HEWI support	0044 1634 258200 (UK), 0049 5691 82-0 (International)
(F) Support HEWI	0033 472 83 09 09 (F), 00 800 4394 4394 (BE, LU, CH appel gratuit, poste fixe)
(NL) HEWI support	00 800 4394 4394 (BE, LU, NL gratis, vaste lijn)
(I) HEWI supportotel	0049 5691 82 0
(E) HEWI soporte	0049 5691 82 0
(PL) HEWI wsparcie	0048 022 330 0052

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(GB) Content	
Article overview	4
Proper use	5
Mounting information	7
Mounting preparation	14 - 15
Mounting instructions	16
Adjusting the seat	17
Brake setting of the armrests	18 - 19

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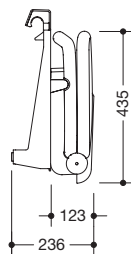
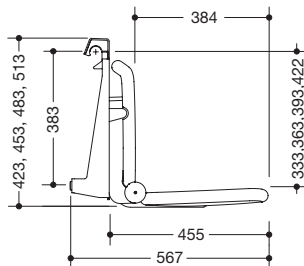
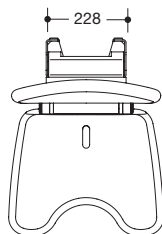
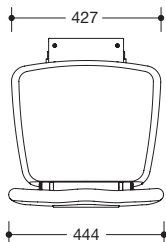
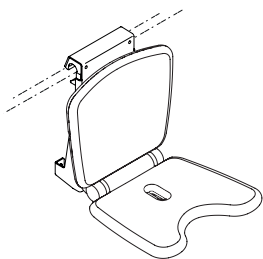
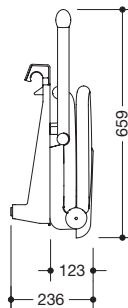
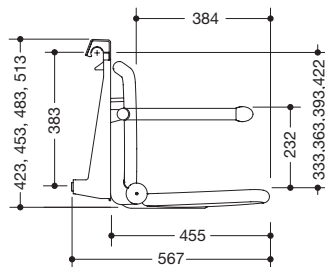
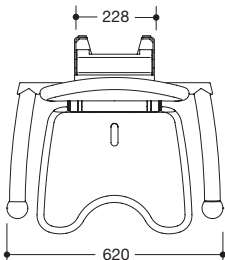
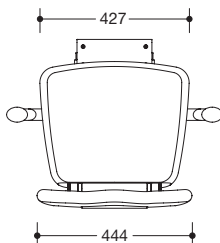
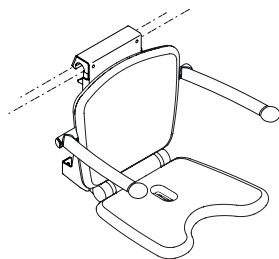
(GB) Proper use in accordance with Directive 93/42/CEE

The HEWI removable hanging seat with lift-up feature is designed to be used indoors, particularly in bathrooms and above all in showers. It offers persons with or without a physical disability the option of sitting down for a shower.

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Safety advice

- Only use the HEWI removable hanging seat in the manner described in the instructions.
- Improper installation/use can cause accidents and injuries.
- The installer must pass on the installation and use instructions to the user and they must be kept for the whole life of the product.

Mounting

- On acceptance the product must be checked for damage; any defects must be reported immediately.
- The product must be stored in a dry place and free from pressure.
- The surface of the seat is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the seat in its packaging whenever possible.
- Mounting only by persons familiar with mounting of the product and appropriate qualifications for their activity.
- Lightweight partition walls must be fitted with timber noggins in places where the rail is fastened and the removable hanging seat's buffer area is located.
- The wall on which the seat is to be installed must be flat, firm and capable of taking the load. Only use fixing materials approved by the wall anchor manufacturer for the intended purpose. In case of doubt, contact HEWI's sanitary products customer service.
- After installing the seat, perform a functional test.

Utilisation

- The HEWI removable hanging is designed for a maximum static working load of 150 kg vertically. The armrests can each withstand a vertical load of 50 kg.
- Do not use this seat for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the seat. In such cases, the seat must be examined by a specialised company work prior to the seat being used again.
- The fact that the seat bends to a certain extent under load is normal. If you consider the seat to bend excessively under load, you can adjust its angle of inclination.
- Do not use the hinged seat for any purpose other than that for which it is intended, e.g. do not use it as gymnastics equipment or as a climbing aid or step.

- When the seat is folded down ensure that there is no part of the body under the product.

Maintenance

The product must be serviced annually.

1. Seat fixing
2. Function and adjustment of the brake unit
3. Secure fit of the spring stop bolt in the seat bracket
4. Function (spring effect) of the spring stop bolt

Care and cleaning instructions

HEWI products are easy care and easy to clean. Frequently, occasional wiping with a damp cloth is sufficient. If using cleaning products the following should be noted:

- Only use cleaning products with a pH value between 6 and 8.
- Cleaning products that contain detergents such as phosphates, soaps and tensides (surfactants) as their active substances can be used.
- Please note and follow the cleaning product manufacturers' instructions (concentration, contact time).
- Do not use alkali, acidic or chlorine-based cleaning products or cleaning products with scouring agent additives
- Do not use any scouring cleaning equipment, e.g. brushes
- Do not apply the cleaner directly to the surface, but always pour it on the cleaning cloth instead.

Disclaimer

HEWI does not accept any liability whatsoever for personal injuries or damage to property that is or has been caused due to incorrect assembly, installation or use, especially due to disregard of the listed warnings and safety instructions.

Disposal

Proper disposal is subject to the applicable national statutory regulations.

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HEWI Heinrich Wilke GmbH