DEUTSCH

ENGLISH

FRANÇAIS

ESPAÑOL

ITALIANO

HEINE Optotechnik GmbH & Co. KG
 Kientalstr. 7 · 82211 Herrsching · Germany
 Tel. +49 (0) 81 52 / 38 - 0
 Fax +49 (0) 81 52 / 38 - 202
 E-Mail: info@heine.com · www.heine.com
 MED 112680 2017-01-31

Produktübersicht / Product overview / Vue d'ensemble du produit / Partes del producto / Informazioni generali sul prodotto

Produktübersicht / Product overview / Vue d'ensemble du produit / Partes del producto / Informazioni generali sul prodotto

DEUTSCH

HEINE Otoskope

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ENGLISH

HEINE Otoscopes

These instructions apply to the following products of the HEINE Otoscope series: BETA200 LED, BETA200, BETA400 LED, BETA400, K180, mini3000® F.O., mini3000® F.O. LED, BETA100, K100, mini3000®.

🔧 Please read and follow these instructions for use of and keep them for future reference.

Intended Use
 HEINE otoscopes are intended for the examination of the ear, the external auditory canal and the tympanum. The instruments have optical magnifying equipment and battery or rechargeable battery powered illumination equipment. The instruments may only be used by qualified medical personnel. Patients can be examined as often as necessary without any further restrictions. The devices are intended for short term treatment and can also be used for instrumentation with non-active medical devices.

⚠️ For U.S. only: Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

⚠️ CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

🔧 NOTE! Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Product overview (see figure)

1 Slit	6 Insufflation bulb
2 Viewing window	7 Insufflation port connector
3 Insufflation port	8 HEINE SANALON® S tips (re-use)
4 Connector plug	9 HEINE AllSpec® tips (single use)
5 HEINE Handle	

Setting up
 To set up the HEINE devices, screw the instrument head into the HEINE battery handle or plug it on the HEINE rechargeable handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:
 White ring = HEINE XHL® 2.5 V bulb
 only for use with the HEINE BETA Battery handle
 Red ring = HEINE XHL® 3.5 V bulb
 Black ring = HEINE LED illumination
 only for use with the HEINE BETA Rechargeable Handle

This does not apply to the otoscope of the mini3000 series. Their lamp voltage is always 2.5 V independently of the handle.

Operation
 The HEINE otoscope may only be inserted into the auditory canal when a tip is fitted.

Place the HEINE tip on the otoscope so that the projection inside the tip fits into the bayonet slit (1). Tighten the tip by twisting it slightly to the right.

The magnifying lens is inside the viewing window (2). To facilitate instrumentation, the viewing window can be swivelled to both sides or folded up.

The otoscopes have a connection port (3) for the optionally available insufflation bulb (6, 7).

The test of tympanic mobility can be carried out when the viewing window is closed using the insufflation bulb. Apply the pressure carefully with the insufflation bulb.

If, after switching on the unit, a sharp drop in brightness or a flashing of the light is noticed, new batteries should be inserted or rechargeable batteries recharged.

🔧 Application duration
 The otoscopes are intended for short term treatment < 1 min with a break between applications of 10 minutes.
 The maximum temperature at the application point at an ambient temperature of 35 °C and in continuous operation is 44.5 °C.

The setting up and operation of the HEINE handles are described in a separate instruction of use.

Hygienic Reprocessing
 Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.
 Classification according to KRINKO: noncritical
 Spaulding Classification USA: noncritical

⚠️ In the event of suspected contamination, carry out hygienic preparation of the instrument.

The reusable tips have to be reprocessed after each use.
 The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the reprocessing agents and procedures named in these instructions for use.
 Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.
 Do not use spray or immersion disinfection, dripping wet or heavily foaming tissues.
 Do not use automated or ultrasonic reprocessing.

Procedure
Otoscopes: The otoscope and the insufflation bulb must be cleaned on the outside manually with a damp cloth and inside with a cotton-wool bud (wipe cleaning and wipe disinfection).

Recommended agents
 Cleaning agent: enzymatic (e.g. neodisher® MediClean)
 Disinfectant agent: alcoholic (e.g. Incides® N)

Reusable tips: The reusable tips must be cleaned and disinfected manually (in an immersion bath) or by machine.

Recommended agents
 Cleaning agent: enzymatic (e.g. neodisher® MediClean)
 Disinfection agent: alcoholic (e.g. rotasept®); agent ortho-phthalaldehyde (e.g. Cidex® OPA)
 The reusable tips can be reprocessed up to 360 cycles.
 Disposable tips (HEINE AllSpec®/HEINE UniSpec®) are for single use only.

Changing the light source

⚠️ Verify that the lamp voltage complies with the supply voltage of the handle.
 Allow the device to cool down before changing the bulb.

BETA200 LED / BETA400 LED / mini3000® F.O. LED
 With these otoscopes, the LED cannot be changed.

BETA200 / BETA400 / K180 / mini3000® F.O.

- Detach the otoscope from the handle and pull the bulb out of the guide tube of the connector plug (4) holding it at the narrow collar.
- Insert the new lamp all the way into the lamp guide.

BETA100 / K100

- Remove the tip from the otoscope and unscrew the lamp. Remove the sleeve from the old lamp.
- Refit the new bulb into the socket.
- Slide the sleeve on as far as it will go. This reduces stray light and reflections from the inside of the specula and should always be fitted.

mini3000
 With this otoscope, swivel the viewing window (2) to the side and pull out the bulb backwards. Insert the new bulb as far as possible into the socket.

Maintenance and Service
 The instruments do not require maintenance or service.
 The expected operational life is 5 years.

General Warnings

⚠️ Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

This product is not allowed to enter or be used in areas with strong magnetic fields e.g. MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not shine light of the instrument directly into the eyes.
 Check the tip before each use for rough surfaces and edges.

In order not to exceed the surface temperature of 41 °C of the application part, keep within the application duration.

General Notes

🔧 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

Disposal

🗑️ The product must be recycled as separated electrical and electronic devices.
 Please observe the relevant state-specific disposal regulations.

Electromagnetic Compatibility
 Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

⚠️ This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.

The use of accessories, converters or cables other than the ones specified by HEINE might lead to increased emission and reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices. If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains following tables

- Guidance and manufacturer's declaration – Electromagnetic immunity
- Technical specification
- Explanation of the used symbols

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Guidance and manufacturer's declaration – electromagnetic emissions		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.		
Emission test	Compliance	Electromagnetic environment – Guidelines
RF emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely that any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. Warning: This device is intended only for use by medical professionals. This is a device of class A CISPR 11 in the domestic environment, this device may cause radio interference, so that it may be necessary in this case, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.
Harmonic Emissions IEC 61000-3-2	not applicable	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Passed	

Guidance and manufacturer declaration - Electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – Guideline
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/ burst IEC 61000-4-4	± 2 kV for mains cables ± 1 kV for input and output lines	± 2 kV for mains cables ± 1 kV for input and output lines	The supply voltage quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV voltage phase – phase, ± 2 kV voltage phase – earth	± 1 kV voltage phase – phase ± 2 kV voltage phase – earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U _r (>95% dip in U _r) for 1/2 period 40% U _r , (60% dip in U _r) for 5 periods 70% U _r , (30% dip in U _r) for 25 periods <		